

**drive**<sup>TM</sup>



## **3-POSITION RECLINER STANDARD & EXTRA WIDE**

Standard Model # D574-CHAR, D574-TAN, D574-CHOC

Extra Wide Model # D574EW-CHAR, D574EW-TAN, D574EW-CHOC


### **IMPORTANT SAFETY GUIDELINES**

PERIODICALLY INSPECT ALL PARTS AND REPLACE AS NEEDED • OCCUPANT WEIGHT SHOULD NOT EXCEED 250 LB FOR STANDARD MODEL AND 500 LB FOR EXTRA WIDE MODEL, CASTER WHEELS SHOULD BE LOCKED DURING OCCUPANT TRANSFER

# ASSEMBLY INSTRUCTIONS:

1. Remove back assembly (C) from unassembled chair and set aside momentarily.
2. Slide the one-piece back assembly (C) into the square tubes of the lower frame assembly (B) so the holes in the backrest line up with the holes located on the horizontal brace to the lower frame assembly (B).
3. Raise recliner Lock Bar (F), adjust position of Back Assembly (C) until Lock Release Button (G) positively engages middle slot lock bar when chair is in storage position.
4. Insert screws through holes of the horizontal brace and tighten all screws with screwdrivers.

**NOTE:** Make sure that the top screws on both sides of the coated square tubing from the back assembly are secure before using recliner.

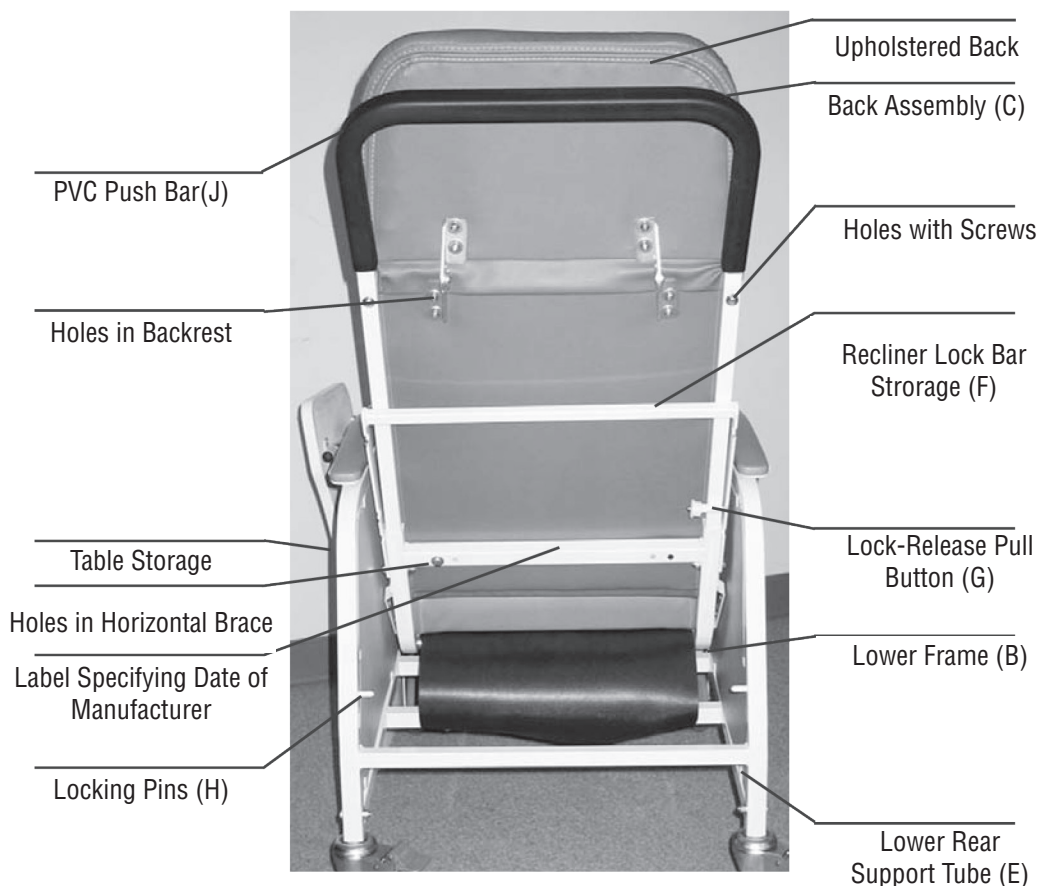

# OPERATING INSTRUCTIONS:

## TO PLACE UNIT IN RECLINING POSITION

### BY OCCUPANT:

Place hands on armrests and push against seat back. Legrest will automatically adjust to each recline position

**NOTE: The table can only be used in upright sitting position. Patient should be transported in reclined position with recliner lock set.**

### BY ATTENDANT:

1. Ensure that the recliner Lock Bar is in stored position. See fig. (1)
2. Stand at either side of the recliner.
3. Grasp armrest with one hand, and PVC push bar (J) with other.
4. Push down on push bar to recline the back to the first recline position. To achieve second recline position, push on push bar again.
5. To raise upright, pull forward on push bar.
6. Set recliner Lock Bar (F).

## RECLINING LOCK OPERATION

### TO LOCK RECLINER

1. Place recliner into desired position.
2. Pull out Lock-Release Pull Button (G) to release Lock Bar (F) from stored position.
3. Rotate recliner Lock Bar (F) down until notches are fully engaged over Locking Pins (H).

### TO UNLOCK RECLINER:

1. Lift recliner Lock Bar (F) up to disengage from locking pins.
2. While holding Lock-Release Pull Button (G) out, rotate recliner lock Bar (F) all the way up until completely against seat back.
3. Release Lock-Release Pull Button (G) to hold recliner Lock Bar in stored position (F).

**NOTE: Do not attempt to change position of chair with recliner Lock Bar engaged. This could cause damage to mechanism.**

# OPERATING INSTRUCTIONS

Articulating headrest may be adjusted to desired position by pulling forward or pushing back.

## OPERATING INSTRUCTIONS FOR BLOW-MOLDED TABLE

### To Operate

1. Underneath each arm, pull down and hold button (fig. 2 A & B) and slide lockout keys forward on both sides (D).
2. Pick up table by circular table arm and slide into receiving tubes in front of each arm.
3. Table can slide on and off by keeping lock/lockout key forward.
4. Table can be locked into one of five positions. Pull down the pull button, hold the pull button and slide the key backwards. Repeat on the other arm and move that key backwards. Table will now be locked.

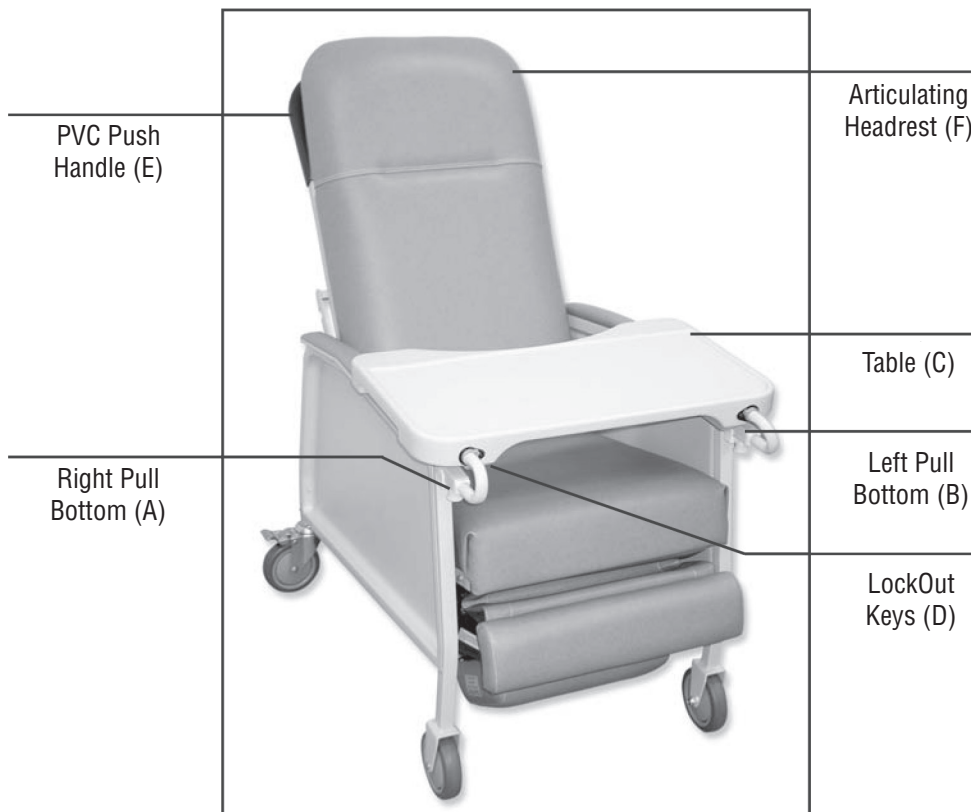


Figure 2

### STORAGE

The blow-molded table stores in the small round tube underneath the left armrest, leaving the tabletop facing out.

### CAUTION:

DO NOT OPERATE RECLINER BY APPLYING FORCE OR PRESSURE TO THE LEGREST. AVOID PLACING HANDS OR LEGS IN CREASES NEAR RECLINER MECHANISM.

# MAINTENANCE & CLEANING

### Every Three Months:

- Check and Clean Casters • Tighten Nuts • Check Brakes on Casters • Check and Clean Lock Recline Mechanism

### For upholstery maintenance and stain removal, the following hints will be helpful:

**Ordinary Dirt:** Wash with warm water and a mild soap or detergent.

**Stubborn Dirt:** Using a soft bristle brush, follow direction above.

**Ground-In Dirt:** Scrub with a mild cleaning solution until clean.

**Chewing Gum:** Scrape carefully: remove with kerosene or naphtha.

**Nail Polish and Remover:** Blot immediately to prevent damage.

**Tars, Asphalt, Creosote:** Remove to prevent staining. Clean with kerosene or naphtha.

**Caution:** When using kerosene or naphtha, do so in a well ventilated area and keep away from fire and flame.

**Paint, Shoe Polish, Heel Marks:** Remove immediately! Use a white cloth dampened in kerosene, naphtha or turpentine. Do not use paint remover or liquid brush cleaners.

**Ball Point Pen:** Blot immediately with a white cloth dampened in water or mineral oil.

**Waxing and Refinishing:** Improper wearability. Use any quality paste wax.

**A diluted solution of 10% (or less) bleach or Clorox may be used when cleaning this chair.**

## Parts For Standard Model

D574BACK-CHAR	Back Only, D574, Charcoal,1/bx
D574BACK-CHOC	Back Only, D574-J, Chocolate,1/bx
D574BACK-TAN	Back Only, D574-R,Tan,1/bx
D574P-1055CHAR	Legrest, D574, Charcoal,1/bx
D574P-1055CHOC	Legrest, D574,Chocolate,1/bx
D574P-1055TAN	Legrest, D574, Tan,1/bx
D574P-1058CHAR	Footrest, D574, Charcoal,1/bx
D574P-1058CHOC	Footrest, D574, Chocolate,1/bx
D574P-1058TAN	Footrest, D574, Tan,1/ea
D574P-1070CHAR	Armrest, D574, Charcoal,1/ea
D574P-1070CHOC	Armrest, D574, Chocolate,1/ea
D574P-1070TAN	Armrest, D574, Tan,1/ea
D574P-SEATLEG-CHAR	Seat with Legrest, D574, Charcoal,1ea
D574P-SEATLEG-CHOC	Seat with Legrest, D574, Chocolate,1ea
D574P-SEATLEG-TAN	Seat with Legrest, D574, Tan,1ea
D574SEAT-CHAR	Seat Only, D574, Charcoal,1/bx
D574SEAT-TAN	Seat Only, D574, Tan,1/bx
D574SEAT-CHOC	Seat Only, D574, Chocolate,1/bx
D574-BBN	Back Release Button, D574,1/ea
D574P-1053	Seat Frame, D574,1/ea
D574P-1027	Front Casters w/ Hardware, D574, 1pr/bx
D574P-1020	Cut Side Panel, D574, Left & Right,1pair
D574P-1003	Hardware Pack, D574,1s et/bx
D574P-1004	Push Bar for D574,1/ea
D574P-1025B	Rear Locking Caster, D574, 1pr/bx
D574P-1035	Recliner Mechanism, D574, Right & Left, 1/bx
D574P-1036	Table Fitting, D574,1/ea
D574P-1037	Washer & Screw, D574 Tray,1/ea
D574P-1038	TrayTable Assembly, D574, 1/cs
D574P-1048	Headrest Adjustment Bracket, D574/D574EW, Left, 1ea
D574P-1060	Footrest Support Bar, D574,1/ea
D574P-1061	Bolt Set, Footrest Support Bar,1/ea
D574P-1064	Slide Tube, Table Tray, D574,1/pair
D574P-BTN	Release Button, Tray, D574,1/ea
D574P-1048-R	Headrest Adjustment Bracket, D574/D574EW, Right, 1ea
D577P-1012	Protecting Panel, D577,1/ea

## Parts For Extra Wide Model

D574-BBN	Back Release Button, D574, 1/ea
D574EW-1003	Hardware Pack, D574, 1set/bx
D574EW-1004	Push Bar for D574,1/ea
D574EW-1010CHAR	Footrest, D574EW, Charcoal, 1/bx
D574EW-1010CHOC	Footrest, D574EW, Chocolate, 1/bx
D574EW-1010TAN	Footrest, D574EW, Tan, 1/bx
D574EW-1011	Footrest Support Bar, D574EW, 1/ea
D574EW-1020	Cut Side Panel, D574, Left & Right,1pair
D574EW-1025B	Rear Locking Caster, D574, 1pair/bx
D574EW-1027	Front Casters w/ Hardware, D574, 1pair/bx
D574EW-1035	Recliner Mechanism, D574, Right & Left,1/bx
D574EW-1036	Table Fitting, D574, 1/ea
D574EW-1037	Washer & Screw, D574 tray, 1/ea
D574EW-1048	Headrest Adjustment Bracket, D574/D574EW, Left,1ea
D574EW-1048-R	Headrest Adjustment Bracket, D574/D574EW, Rght,1ea
D574EW-1050	Lock Bar for Back, D574,1/ea
D574EW-1061	Bolt Set, Footrest Support Bar, 1/ea
D574EW-1064	Slide Tube, Table Tray, D574,1/pair
D574EW-1065	Tray Table Assembly, D574EW, 1/cs
D574EW-1070CHAR	Armrest, D574EW, Charcoal, 1/ea
D574EW-1070CHOC	Armrest, D574EW, Chocolate, 1/ea
D574EW-1070TAN	Armrest, D574EW, Tan, 1/ea
D574EWBACK-CHAR	Back, D574EW-Char, 1/bx
D574EWBACK-CHOC	Back, D574EW-Choc, 1/ea
D574EWBACK-TAN	Back, D574EW-R, Tan, 1/bx
D574EWSEAT-CHAR	Seat w/ Legrest, D574EW-Char,1/bx
D574EWSEAT-CHOC	Seat w/ Legrest, D574EW-Choc, 1/ea
D574EWSEAT-TAN	Seat w/ Legrest,D574EW-Tan,1/bx
D574P-BTN	Release Button, Tray, D574,1/ea
D577P-1012	Protecting Panel, D577,1/ea
D574EW-1017N	Frame, D574EW, 1/ea
D574EW-1065	Tray Table



SUNGO Europe B.V.  
Olympisch Stadion 24, 1076DE  
Amsterdam, Netherlands



Port Washington, NY 11050  
Toll free: 877.224.0946  
Phone: 516.998.4600  
www.drivemedical.com



**WARNING:** This product can expose you to chemicals, including tris(1,3-dichloro-2-propyl) phosphate (TDCPP), which is known to the State of California to cause cancer. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).