

# lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China


universal crutch/walker  
platform attachment

drive  
MEDICAL DESIGN & MANUFACTURING

[www.drivemedical.com](http://www.drivemedical.com)

## instructions

1. Determine which side of the platform attachment will be used.
2. Attach armrest and handle to vertical platform post using the supplied bolts and adjustment.
3. Place both platform clamps on the vertical platform post with the longer clamp on first (top).
4. Loosely attach the bottom (smaller) clamp to the lower horizontal walker crossbar.
5. Loosely attach the top (large) platform clamp to either the upper horizontal walker crossbar or over the top bar handle.
6. Adjust platform post to desired height.
7. Tighten both clamps carefully adjusting both sides of clamps equally.
8. Adjust arm support and handle to increase patient comfort and support. Tighten all adjustment knobs to ensure patient safety.

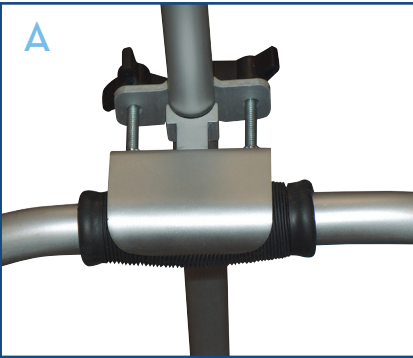


**A.** Larger upper bracket fits over the handle of the walker.



**B.** Smaller bottom bracket is placed over the bottom cross brace of the walker.

**C.** Tighten both brackets securely when desired height is achieved.

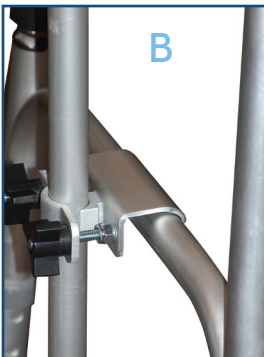

Item # 10105-1 Shown

