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aluminum crutches

Push-Button/Adjustable Fully Assembled

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drive
MEDICAL DESIGN & MANUFACTURING

10400-8 Adult

Patient Height
5'2" to 5'10"

10400-8 Adulto

Altura del
paciente
5'2" to 5'10"

10400-8 Adulte

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10401-8 Youth

Patient Height
4'6" to 5'2"

10401-8 Joven

Altura del
paciente
4'6" to 5'2"

10401-8 Enfant

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10402-8 Tall

Patient Height
5'10" to 6'6"

10402-8 Alto

Altura del
paciente
5'10" to 6'6"

10402-8 Grande taille

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CAUTIONS:

- Use after proper training under the direction of your Doctor or Physical Therapist.
- Replace Crutch Tips if worn.
- Keep Hand Grip Wing Nut tightened.
- Use extra caution on wet and slippery surfaces.

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For maximum fit and assembly, see instructions below:

1. **Height:** Adjust height by pressing both push-buttons, rotating foot-piece, and pulling down until desired patient height is obtained. To help in doing this, the foot-piece has a circular ring marked to indicate location of push-button. Rotate foot-piece again until both push buttons snap back into place.
2. **Handgrip:** Adjust handgrip to fit an approximate 10" bend at patient elbow.
3. **Safety Check:** With crutches positioned per diagram below, confirm patient can lean forward comfortably while resting on the underarm supports. Patient should not have to stand uncomfortably erect. Ensure both push-buttons are firmly in place and handgrip nuts are tightened securely.

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