drive™

www.drivemedical.com

Rev.2.04.28.14

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item # 11148N-4

EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

drive™

assembly instructions and precautions

STEP 1

STEP 2

STEP 3

1. Remove commode from packaging.
2. Unfold commode and stand upright.
3. Adjust the sides of commode frame perpendicular to the back frame.
4. Attach front cross brace by depressing brass push pins and sliding brace into side frame socket. Attach on both sides. Be sure brass push pins "snap" through holes.
5. Attach back brace by depressing brass push pins and sliding back brace into receptacle located on back of cross brace. Be sure brass push pins "snap" through holes.
6. Lower commode bucket into bracket and attach lid onto back of frame.
7. To unfold, depress push pins on back support and remove. Depress push pins on front frame and slide frame from side.

adjusting leg height

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per illustration.
2. To be sure legs are locked into place by pushing down on or by pulling legs.

FIGURE 1

additional precautions

1. Toilet seat must be in down position before use. Failure to do so may result in breakage of commode and possible injury.
2. Do NOT install commode without first reading and understanding this assembly and instruction sheet. If you can not understand these instructions, contact a healthcare dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.
3. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
4. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved (see Figure 1).
5. Ensure ALL screws, nuts and/or bolts are tightened.
6. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
7. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
8. Users with limited physical strength should be supervised or assisted while using commode.

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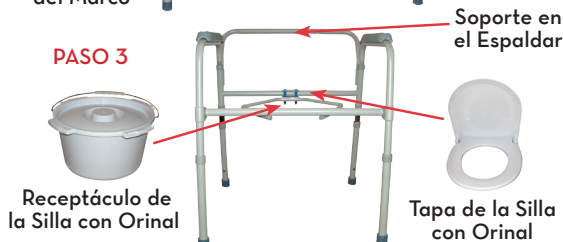
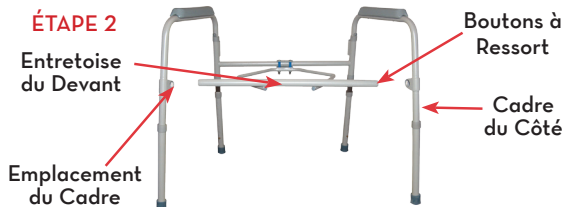
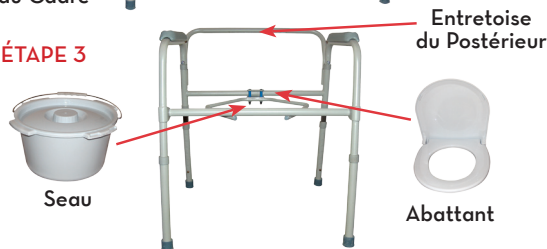
lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2014 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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The logo for Drive Medical, featuring the word "drive" in a bold, lowercase, blue sans-serif font. A small yellow square is positioned above the letter 'i'. A trademark symbol (TM) is located to the upper right of the word.

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A large, stylized version of the Drive logo, oriented vertically. The word "drive" is written in white, lowercase, bold letters against a dark blue background. A small yellow square is positioned above the letter 'i'. A trademark symbol (TM) is located to the upper right of the word.

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© 2014 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.