

## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2005 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

mayo instrument stand  
item # 13035




EU Authorized Representative  
Drive Medical LTD  
Ainley's Industrial Estate  
Elland, West Yorkshire,  
United Kingdom HX5 9JP


drive  
WWW.DRIVEMEDICAL.COM

# assembly instructions

1. Verify that all the parts are in the box.

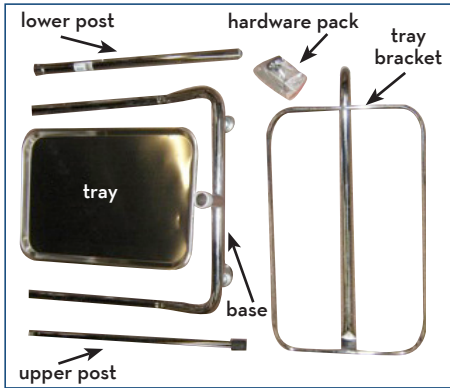

Parts

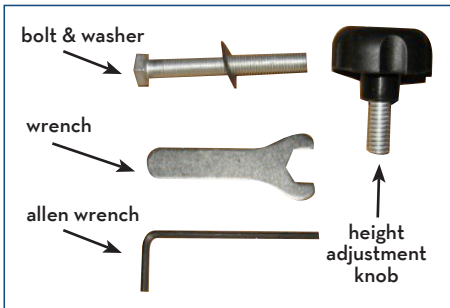

Hardware Pack

2. Find the end of the lower post that has a threaded insert. (figure 2)



figure 2

3. Insert this end into the base of the unit with the curve of the post matching the curve of the base. (figure 3)

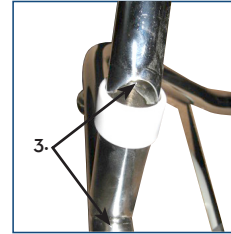


figure 3

4. Insert the bolt with washer through the bottom of the base and hand tighten into the lower post. (figure 4)

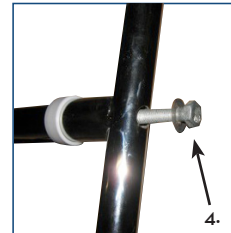


figure 4

5. Tighten the bolt with the wrench provided in the hardware pack. (figure 5)

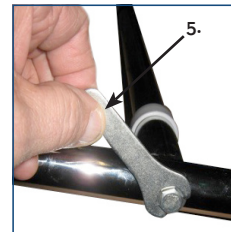


figure 5

6. Insert height adjustment knob into the threaded hole on the top of the lower post. (figure 6)

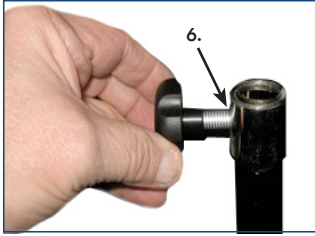


figure 6

7. Remove the protective cap from the lower part of the upper post. (figure 7)

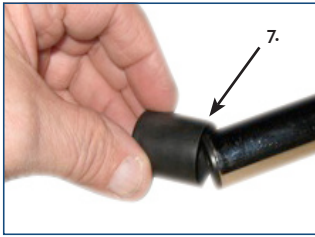


figure 7

8. Insert this end of the upper post into the upper portion of the lower post. Tighten the height adjustment knob. (figure 8)

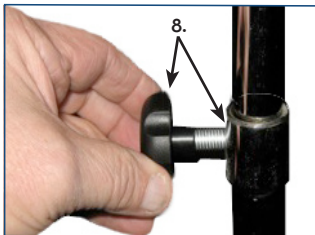


figure 8

9. Remove the protective cap from the tray bracket. (figure 9)

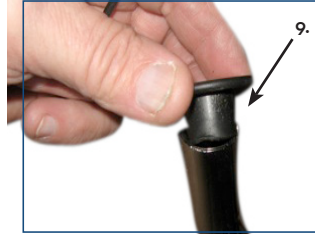


figure 9

10. Put the tray bracket onto the upper post, align the half circle cut out with the allen set screw. (figure 10)

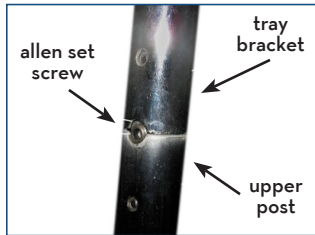


figure 10

11. Tighten the allen set screw with the allen wrench provided with the hardware pack. (figure 11)

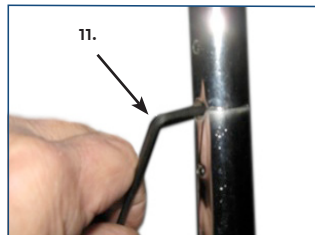


figure 11

12. Put the tray on top of the tray bracket.