

## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

## 3 panel privacy screen

item # 13508

drive  
MEDICAL DESIGN & MANUFACTURING

[www.drivemedical.com](http://www.drivemedical.com)

# assembly instructions

1. Remove screen and caster box from carton.
2. Remove nuts and bolts and casters from box.
3. Place screen on it's side.
4. Slide a plastic cap over the leg.
5. Unscrew nuts from bolts and position one of the washers on the bolt.
6. Align one of the casters with the holes in the leg of the screen.
7. Insert bolt through the holes of the caster and screen leg. Be sure to position a washer in between the bolt head and the caster leg.
8. Place another caster leg over the end of the bolt that is protruding form the hole.
9. Place a washer over the end of the bolt.
10. Place a nut over the end of the bolt and use the provided allen wrench to tighten the bolt.
11. Repeat for other screen leg.
12. Make sure all bolts are firmly tightened.
13. Slide plastic cap down to cover top of caster legs.

