



Reusable Nebulizer Kit

REF 18254

LOT

Contains : one (1) ea

Includes: Mouth piece, T piece, corrugated reservoir tube,
Medication Cup, 7' tubing.

Designed for respiratory therapy

Material: polypropylene, polyethylene, polyvinyl chloride, polycarbonate



Caution

- For the best performance, replace every 6 months from first use.
- Store away from heat and light.

Instructions for use:

1. Attach T Piece to the medication cup.
2. Attach mouthpiece and corrugated reservoir tube to T piece.
3. Connect the 7' tubing to the medication cup and the nebulizer machine.
4. Be sure connections are secured before turning on the nebulizer machine.

Warnings

1. A Nebulizer Kit must only be used by persons who have received adequate training and instruction from qualified medical personnel.
2. Inappropriate use of these products may result in harm to the patient.
3. Nebulizer concentration is affected by the patient's tidal volume.
4. Do not use the neb kit or a nebulizer machine in the presence of an open flame, as ignition is a remote possibility with high concentrations of medication present.
5. Be sure connections are secure at all times.
6. Do not sterilize.

Manufactured for:

Drive Medical

99 Seaview Boulevard, Port Washington, NY 11050

Tel: 516-998-4600 Fax: 516-998-4601

www.drivemedical.com



NOT MADE WITH
NATURAL RUBBER LATEX