


# CANISTER KIT

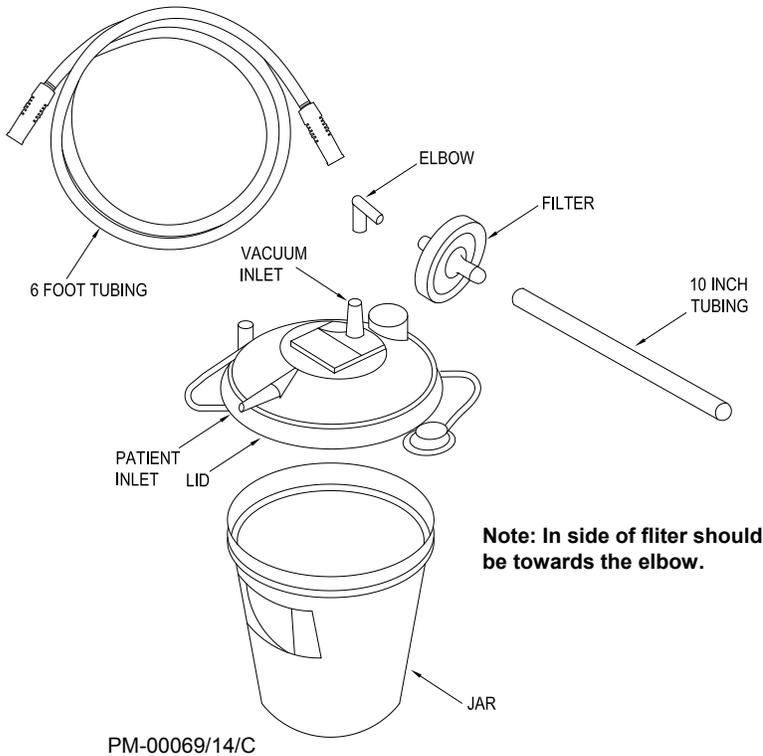


## Setup Instructions

1. Place lid on jar and press it on tight.
2. Connect elbow, filter, and tubing together.
3. Connect 6 foot tubing to the patient inlet on the lid.
4. Connect elbow to the inlet on the lid.
5. Be sure that the collection jar lid is pushed on tight and that all connections are secure and without leaks before using.
6. Keep containers filled with fluid up right at all times.

**drive**<sup>™</sup>

Distributed by:  
Drive Medical Design & Manufacturing  
99 Seaview Boulevard  
Port Washington, NY 11050  
Toll Free: 877-224-0946  
www.drivemedical.com

# CANISTER KIT

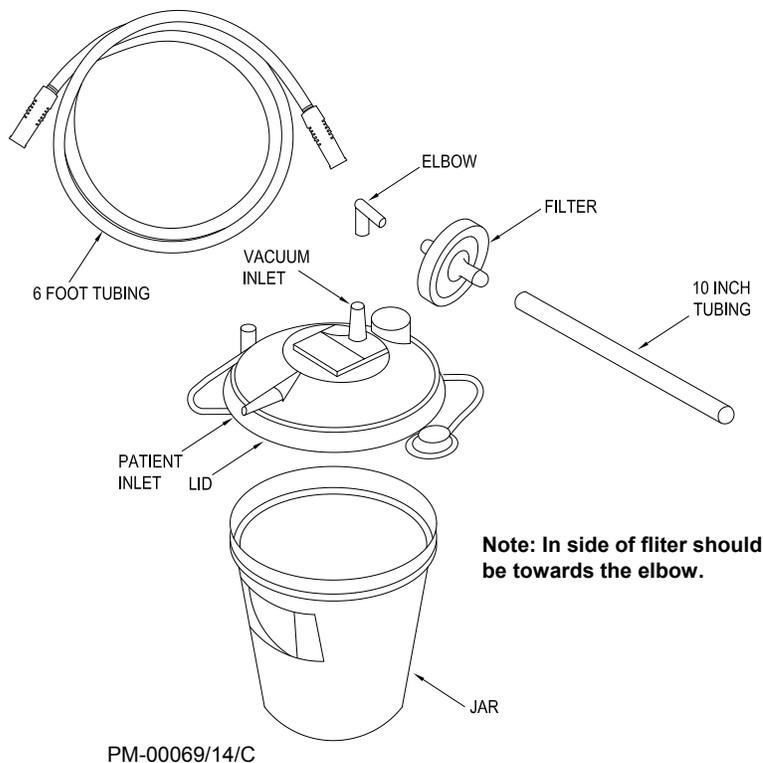

## Setup Instructions

1. Place lid on jar and press it on tight.
2. Connect elbow, filter, and tubing together.
3. Connect 6 foot tubing to the patient inlet on the lid.
4. Connect elbow to the inlet on the lid.
5. Be sure that the collection jar lid is pushed on tight and that all connections are secure and without leaks before using.
6. Keep containers filled with fluid up right at all times.

**drive**<sup>™</sup>

Distributed by:  
Drive Medical Design & Manufacturing  
99 Seaview Boulevard  
Port Washington, NY 11050  
Toll Free: 877-224-0946  
www.drivemedical.com