MOIST/DRY HEATING PAD
ALMOHADILLA TÉRMICA HÚMEDA/SECA
COUSSIN CHAUFFANT HUMIDE OU SEC

RTLAGEF-HP-STD
RTLAGEF-HP-LG

driveTM

INSTRUCTION GUIDE
GUÍA DE INSTRUCCIONES
GUIDE D'INSTRUCTIONS

IMPORTANT SAFETY INSTRUCTIONS

DO NOT DESTROY

DANGER- to reduce the risk of burns, electric shock and fire, this product must be used in accordance with the following instructions:

1. Read all instructions.
2. **ALWAYS UNPLUG THE UNIT FROM THE ELECTRICAL OUTLET IMMEDIATELY AFTER USE.**
3. Do not use while sleeping.
4. Do not use on an infant.
5. This pad is not to be used on or by any individuals with certain disabilities, sleeping or unconscious person, or a person with poor blood circulation unless carefully attended.
6. Do not use on areas of sensitive skin.
7. Burns can occur regardless of control setting, check skin under pad frequently.
8. Never use pad without cover in place.
9. Do not use pins or other metallic means to fasten this pad in place.
10. Do not sit on or crush pad - avoid sharp folds.
11. Never pull this pad by the supply cord.
12. Do not use the cord as a handle.
13. Carefully examine inner core before each use. Discard the pad if inner covering shows any sign of deterioration.
14. Do not tamper with this pad in any way. There are no user serviceable parts.
15. Do not use the pad with liniment, salve or ointment preparations that contain heat-producing ingredients as skin burns could result.
16. Do not use in an oxygen atmosphere.
17. Loop cord loosely when storing, tight wrapping may damage cord and internal parts.

SAVE THESE INSTRUCTIONS

HEATING PAD COMPONENTS

- Washable Cloth Cover (Figure 1). (Wash in luke warm water and let dry)
- Heating Pad (Figure 2)
- Absorbent Sponge (Figure 3)

Figure 1

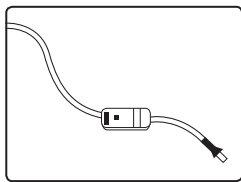
Figure 2

Figure 3

CAUTIONS

- The Cloth Cover must always be used with the Heating Pad. An Absorbent Sponge is provided for moist heat application.
- This heating pad should not be used in the following cases: if you have diabetes, tuberculosis, benign or malignant tumors, phlebitis and thrombosis, hemorrhages, inflammations of the skin, undetermined calf pain or joint pain, or if you have poor circulation. Consult a physician before using if you have any kind of fungal infection, swelling, fracture or persistent pain.
- **Consult your physician should you have any therapeutic questions regarding use of this heating pad.**
- Do not use on any unexplained pain, or swollen muscles, or following a serious muscular injury before consulting a physician.
- Keep away from children.
- Discontinue use of product if pain increases or swelling occurs after application.
- Do not use while bathing or showering. Use only in a dry area.
- Consult your doctor before using in case of illness or medical condition, including but not limited to pacemaker use, pregnancy or cancer.
- If you experience pain in a muscle or joint for a prolonged period of time, discontinue use and consult your doctor. Persistent pain could be a symptom of a more serious condition.

- Never use any heating products on open wounds, discolored areas, or any area of the body that is swollen, burned, bruised, inflamed or where skin eruptions, ulcerated sores or varicose veins are present.
- This product should never be used by any individual suffering from any ailments that would limit or altogether inhibit the user's capacity to feel or have any sensation in the body part that this product is being used for.
- Use of this product should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your physician.
- Ensure that your hands are dry when operating the controls or removing the plug.

HOW TO ASSEMBLE

To assemble the heating pad, insert the heating pad completely into the cloth cover with the electric cord extending out from the cloth cover (Figure 4). If moist heat is desired, follow instructions in the "Moist Heat Treatment" section.

Figure 4

TO OPERATE

To operate the heating pad, first plug the cord into a standard 120V electrical outlet and slide switch to the "ON" position. The indicator light will illuminate to signal the heating pad is "ON". Slide the switch to the "OFF" position when done. **For maximum benefit, use should not exceed 20 minutes at one time.**

This appliance has a polarized plug (one blade is wider than the other). As a safety feature, this plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. Do not attempt to defeat this safety feature.

MOIST HEAT TREATMENT

For your safety, never immerse the electric heat wrap in water or any liquid. Make sure the heating pad is unplugged. Wet the absorbent sponge and squeeze out the excess water. Insert the damp sponge flat between the heating pad and the cloth cover. Then use the heating pad in a normal fashion. After use, dry the cloth cover, absorbent sponge and heating pad before storing.

3 YEAR LIMITED WARRANTY

Your Drive brand product is warranted to be free of defects in materials and workmanship for 3 years of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This 3 Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2012 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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© 2013 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.