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# Compartment Style Compressor Nebulizer

## INSTRUCTIONS

### INDEX

1. Introduction .....	Page 2
2. Product Identification .....	Page 3
3. Important Safeguards .....	Page 3
4. Operating Your Nebulizer .....	Page 4
5. Cleaning .....	Page 5
6. Replacement Parts & Optional Accessories.....	Page 6
7. Specifications.....	Page 7
8. Warranty .....	Page 8



### 1. INTRODUCTION

Thank you for purchasing the Airial™ Compartment Style Compressor Nebulizer. It is a compact medical device designed to efficiently deliver physician prescribed medication to the bronchial lung passages. With proper care and use, it will provide you with many years of reliable treatment.

This product is developed for the successful treatment of asthma, allergies and other respiratory disorders. It creates a stream of air that travels through a clear tube to the nebulizer. When air enters the nebulizer, it will convert the prescribed medication into aerosol mist for easy inhalation.

Your Compartment Style Compressor Nebulizer should be used under the supervision of a licensed physician and/or a respiratory therapist. We encourage you to thoroughly read this guidebook to learn about the features of this product. Any use of this product other than its intended use is not recommended.

## 2. PRODUCT IDENTIFICATION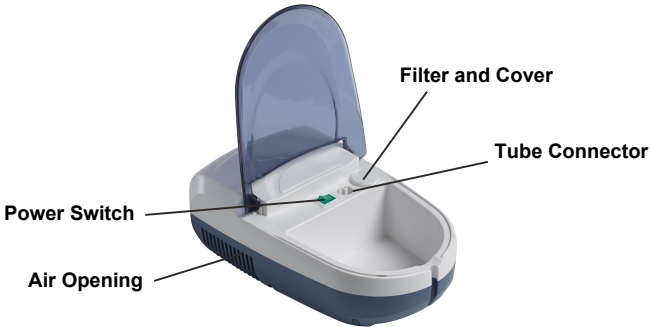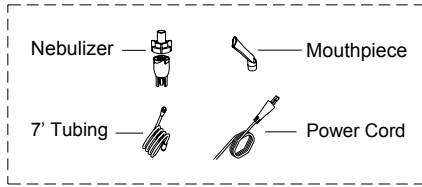

Model #	Description
MQ6000	Airial™ Compartment Style Compressor Nebulizer
MQ6000EU	Airial™ Compartment Style Compressor Nebulizer with Disposable Neb Kit, 230V
MQ6000SA	Airial™ Compartment Style Compressor Nebulizer with Disposable Neb Kit, 230V with UK Plug

## 3. IMPORTANT SAFEGUARDS

**Note:** Read all instructions carefully before use.

The following basic precautions are needed when using an electrical product:

**Caution:** Failure to read and observe all precautions could result in personal injury or equipment damage.

### Product cautions:

1. To avoid electrical shock: Keep the unit away from water. Do not immerse the power cord or the unit in liquid. Do not use while bathing. Do not reach for a unit that has fallen into water – immediately unplug the unit.
2. Never operate the unit if it has any damaged parts (including power cord), or if it has been dropped or submersed in water. Promptly send it to a service center for examination and repair.
3. The unit should not be used where flammable gas, oxygen, or aerosol spray products are being used.
4. Disconnect the unit from the electrical outlet before cleaning, filling and after each use.
5. Do not use other attachments unless recommended by manufacturer.

## Operating Cautions:

1. Only connect this product to an appropriate voltage outlet for your model.
2. Do not run this product unattended.
3. Never operate this unit if it has any damaged parts (including a power cord), or if it has been dropped or submerged in water. If for any reason this unit does not work properly, promptly return it to an authorized service center for repair.
4. If any abnormalities occur, discontinue use immediately until the unit has been examined and repaired.
5. Always unplug the product immediately after use.
6. Never block the air openings of the main unit or place it where the air openings may be obstructed.

## Storage Cautions:

1. Do not store the unit under direct sunlight, high temperature or humidity.
2. Keep the unit out of reach of small children.
3. Always keep the unit unplugged while not in use.


## Cleaning Cautions:

1. Do not immerse the unit in water. It may damage the unit.
2. Disconnect the unit from the electrical outlet before cleaning.
3. Clean all necessary parts after each use as instructed in this guidebook.

## 4. OPERATING YOUR NEBULIZER

**Note:** Prior to initial operation, this product should be thoroughly cleaned.

1. Place your Compressor Nebulizer on a flat and stable surface. Be sure that you can easily reach the controls when you are seated.
2. Gently twist the top part of the nebulizer cup counter-clockwise to disassemble the nebulizer cup.
3. Fill the bottom section of the nebulizer cup with the medication prescribed by your physician. Be sure that the cone is put inside the bottom section.
4. Gently twist the top part clockwise to reassemble the nebulizer cup. Be sure the two sections fit well.
5. Attach one end of air tube to the base of the nebulizer cup.
6. Attach the other end of air tube to the air tube connector located on the top of the compressor.
7. Attach mouthpiece or mask per your choice to the top section of the nebulizer cup.
8. Plug the power cord into an appropriate electrical outlet. Make sure at this stage, the power switch is at "OFF" status.
9. Press the power switch to "ON" status to begin your prescribed treatment.

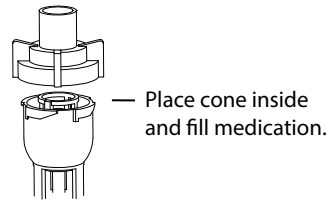**Important:**

The compressor motor has a thermal protector, which will shut off the unit before the unit is overheated. When the thermal protector shuts the unit off, please:

- a. Switch off the unit.
  - b. Unplug the unit from the electrical outlet.
  - c. Wait 40 minutes for the motor to cool down before another treatment.
  - d. Make sure the air openings are not obstructed.
10. When treatment is finished, shut off the unit and unplug it from the electrical outlet.

**5. CLEANING**

It is recommended that the nebulizer, mouthpiece and the mask (or any other patient accessory interface that is used) be thoroughly cleaned with hot water after each use and cleaned with a mild detergent after the last treatment of the day. If your physician or respiratory therapist specifies a different cleaning procedure, follow their instructions.

**Caution:**

Before cleaning, make sure the power cord is disconnected from the main outlet and the unit.

**Rinsing (after each treatment)**

1. Disconnect the air tube, nebulizer cup, mouthpiece and mask.
2. Gently twist the nebulizer cup to open it.
3. Rinse the nebulizer cup, mouthpiece and mask with water.
4. Dry them with clean soft towel or let them air dry.
5. Reassemble the nebulizer cup when completely dry and put these parts in a dry, sealed container.

## **Disinfecting**

Please follow the following steps to disinfect your nebulizer unless otherwise specified by your physician. It is suggested that the unit is disinfected after the last treatment of the day.

1. Use one part white vinegar with three parts distilled water. Make sure mixed solution is enough to submerge the nebulizer, mouthpiece and mask.
2. Complete Rinsing Steps 1-3.
3. Wash nebulizer cup, mouthpiece and mask in warm water and a mild detergent. Then wash them in hot tap water.
4. Submerge these parts in the vinegar and water solution for thirty minutes.
5. Complete Rinsing Steps 3-5.

## **Cleaning the Compressor**

1. Wipe daily with a damp cloth.
2. Do not use any powdered cleaners or soap pads, which may damage the finish.

## **Filter Change**

1. Do not use cotton or any other materials. Do not wash or clean the filter. Only use filters supplied by your distributor. And do not operate without a filter.
2. Changing procedure:
  - A. Remove the filter cover.
  - B. Replace the used filter with a new filter.
  - C. Replace the filter cover.

## **6. REPLACEMENT PARTS AND ACCESSORIES**

Contact your healthcare distributor for replacement parts or accessories.

## 7. SPECIFICATIONS

Power	AC 120V, 60Hz (MQ6000) AC 230V, 50Hz (MQ6000EU, MQ6000SA)
Power Consumption	65W (MQ6000) 55W (MQ6000EU, MQ6000SA)
Medication Capacity	5ml
Particle Size	0.5 to 10 µm
MMAD	4 µm
Sound Level	Around 60 dBA
Maximum Medication capacity	13 ml (cc)
Average Nebulization Rate	0.2ml/min
Maximum Pressure	30 Psig or greater
Operating Pressure Range	10 psig or greater
Compressor Free Air Flow	8 lpm or greater
Operating Temperature Range	10°C to 40°C (50°F to 104°F)
Operating Humidity Range	10% to 95% RH
Storage Temperature Range	-20°C to 70°C (-4°F to 158°F)
Storage Humidity Range	10% to 95% RH
Dimension (L x W x H)	300 mm x 180 mm x 115 mm (11.8" x 7.1" x 4.5")
Weight	4.5 lbs (2.1kg)
Standard Accessories	Nebulizer, Air Tube, Mouthpiece, Filters (5 PCs)

-  - Protection against electric shock
-  - Class II Equipment
-  - Type BF applied parts  
Mouthpiece and mask

Protection against harmful ingress of water  
- IPX0 (Ordinary)

Degree of safety in the presence of flammable anesthetics or oxygen  
- No AP/APG  
(Not suitable for use in the presence of flammable anesthetics or oxygen)

Mode of Operation—20 min. on & 40 min. off

## **8. WARRANTY**

Your Drive brand product is warranted to be free of defects in materials and workmanship for five years of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This five year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as filters, rubber accessories, and rubber grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive Dealer.

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