


770-406 AquaSense® Transfer Bench

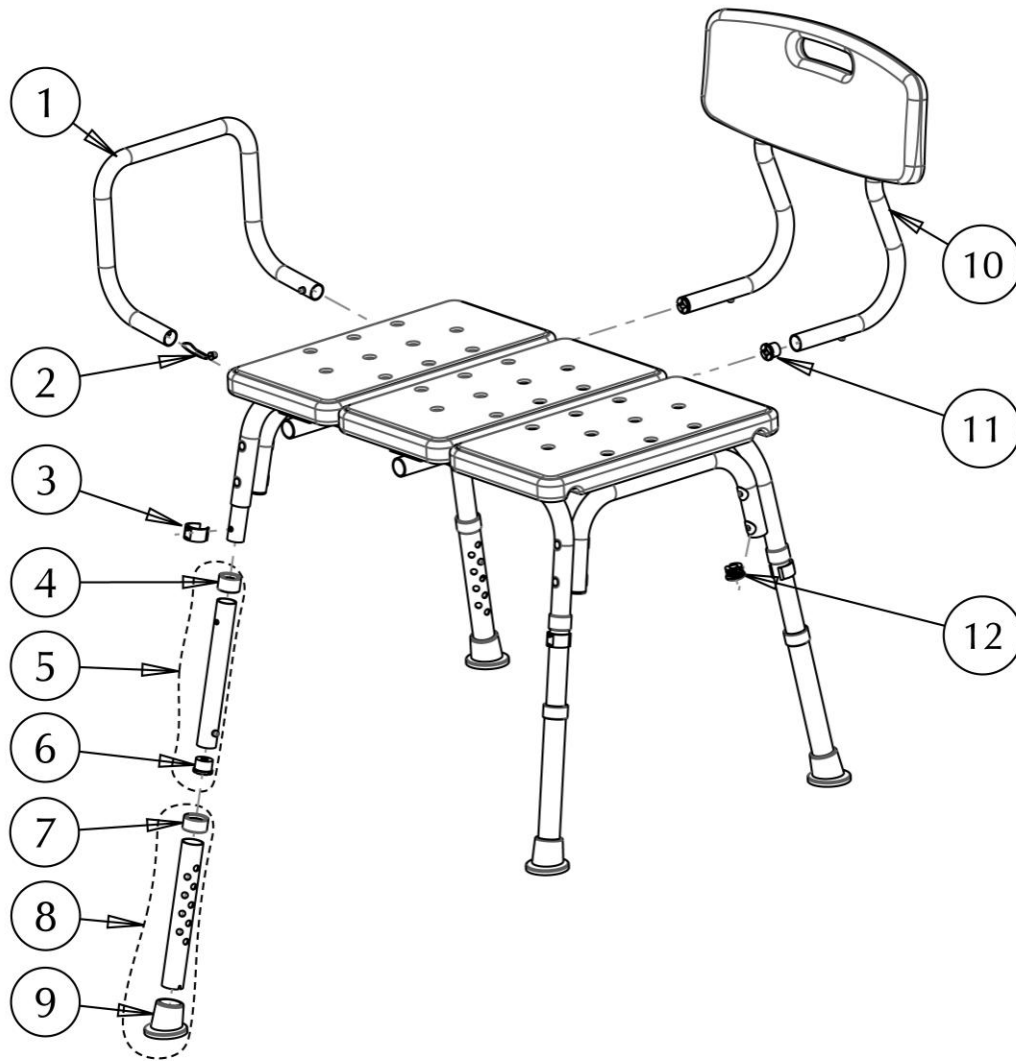

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by
1.	B08-201	Armrest Bar (Aluminum Tube 22.2mm & 2 Spring Pins)	1 EA	1 EA
2.	B08-301	Spring Pins - 2 ea / set	5 ST	1 ST
3.	A08-246	E-Clip for 25.4mm Tube, Aqua-Blue - 2 ea / set	3 ST	1 ST
4.	A08-262	Anti-Rattle for 25.4mm Tube, Aqua-Blue - 4 ea / set	1 ST	1 ST
5.	B08-081	Connection Tube w/Hardware (Aluminum Tube \varnothing 25.4x200mm, Blue Anti-Rattle, Spring Pin & Black Anti-Rattle) - 4 ea / set	1 ST	1 ST
6.	A08-272	Anti-Rattle for 25.4mm Tube, Black - 4 ea / set	1 ST	1 ST
7.	A08-263	Anti-Rattle for 28.4mm Tube, Aqua-Blue - 4 ea / set	1 ST	1 ST
8.	B08-071	Extension Leg (Aluminum Tube 28.4 diameter x 200 mm long, Blue Anti-Rattle and Blue Tip) - 4 extension legs / set	1 EA	1 EA
9.	A08-452	Rubber Tip for \varnothing 28.4mm Tube, Aqua-Blue - 4 ea / set	1 ST	1 ST
10	B08-052	Backrest Complete (White Blow-Molded Plastic Back, 2 Aluminum Tubing, 2 Spring Pins, 2 Self-threading Screws M5x15mm & 2 Anti-Rattle White)	1 EA	1 EA

770-406**AquaSense® Transfer Bench**

11.	A08-271	Anti-Rattle for 22.2mm Tube, White - 2 ea / set	1 ST	1 ST
12.	A08-421	End Plug for dia=25.4mm Tube, Grey - 4 ea / set	1 ST	1 ST