



**700-869 (MIDNIGHT-BLUE)  
HUGO TRANSPORT CHAIR 19" WIDE**

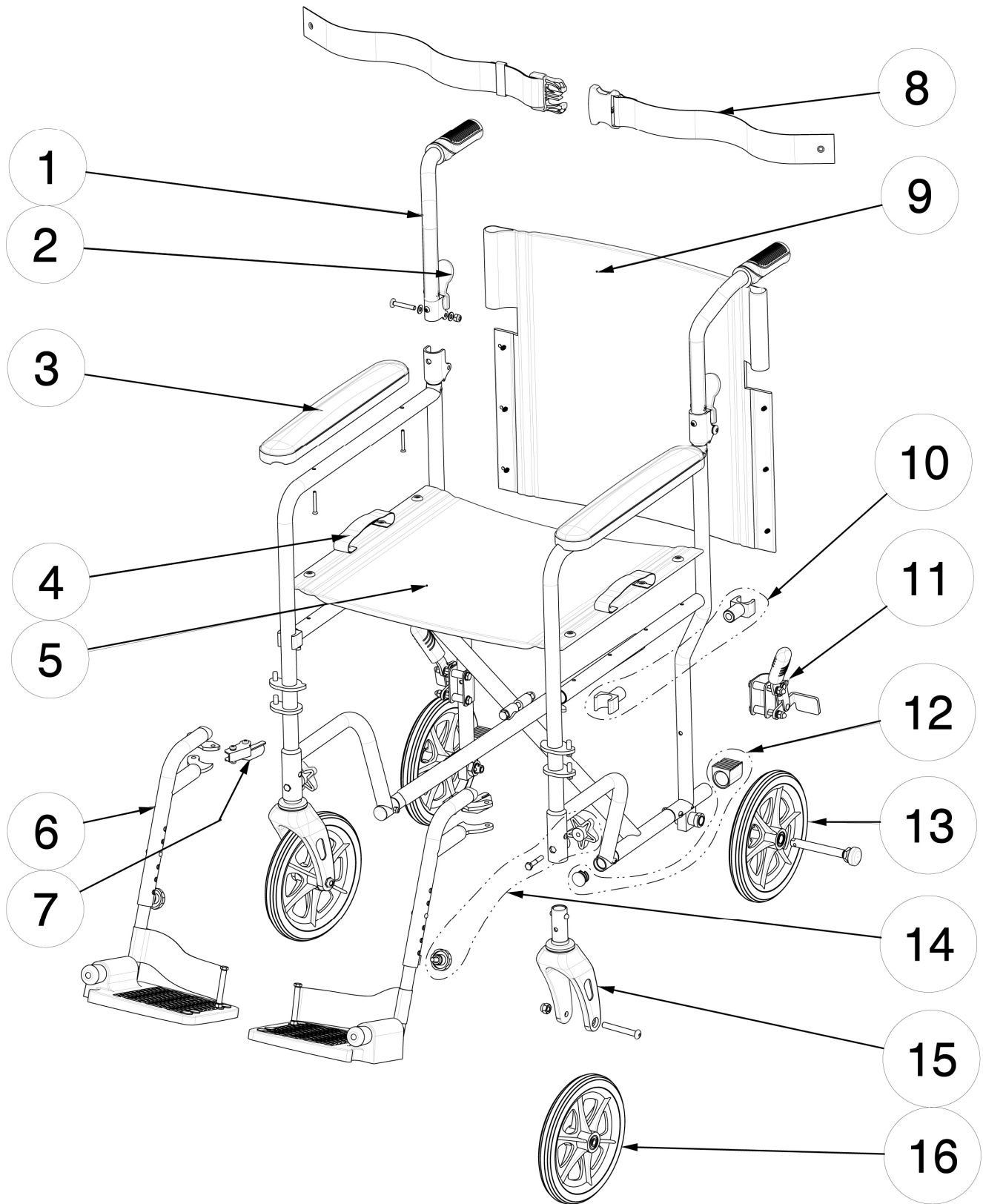

**700-869 (MIDNIGHT-BLUE)  
HUGO TRANSPORT CHAIR 19" WIDE**

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	C48-00076	Blue Folding Handle & Quick Fold Mechanism - L & R	1	1	PR
2.	C48-00077	Handle Release Lever, Black - 2 ea / set	1	1	ST
3.	C48-00078	Full Black Nylon Arm Pad & Hardware – 2 ea / set	1	1	ST
4.	C48-00061	Seat Strap - 2 ea /set	1	1	ST
5.	C48-00079	Black Nylon Seat Upholstery 19"x15.5" & Hardware	1	1	EA
6.	C48-00081	Blue Swing-Away Footrest - L & R	1	1	PR
7.	C48-00082	Black Footrest Release Mechanism - L & R	1	1	PR
8.	C48-00066	Safety Belt, Black	1	1	EA
9.	C48-00084	Hugo Black Nylon Back Upholstery 19"x16" & Hardware	1	1	EA
10.	C48-00069	2 Front & 2 Rear Black Seat Guides	1	1	ST
11.	C48-00070	Wheel Lock Assembly - L & R	1	1	PR
12.	C48-00087	2 Front + 2 Rear End Plugs	1	1	ST
13.	C48-00085	8" Black Rear 6 Spokes Wheel & Hardware	2	1	EA
14.	C48-00086	Set of Knobs & Hardware – 6 ea / set	1	1	ST
15.	C48-00088	Front Fork & Hardware	2	1	EA
16.	C48-00089	8" Black Front 6 Spokes Wheel & Hardware	2	1	EA


### 700-869 (MIDNIGHT-BLUE) HUGO TRANSPORT CHAIR 19" WIDE

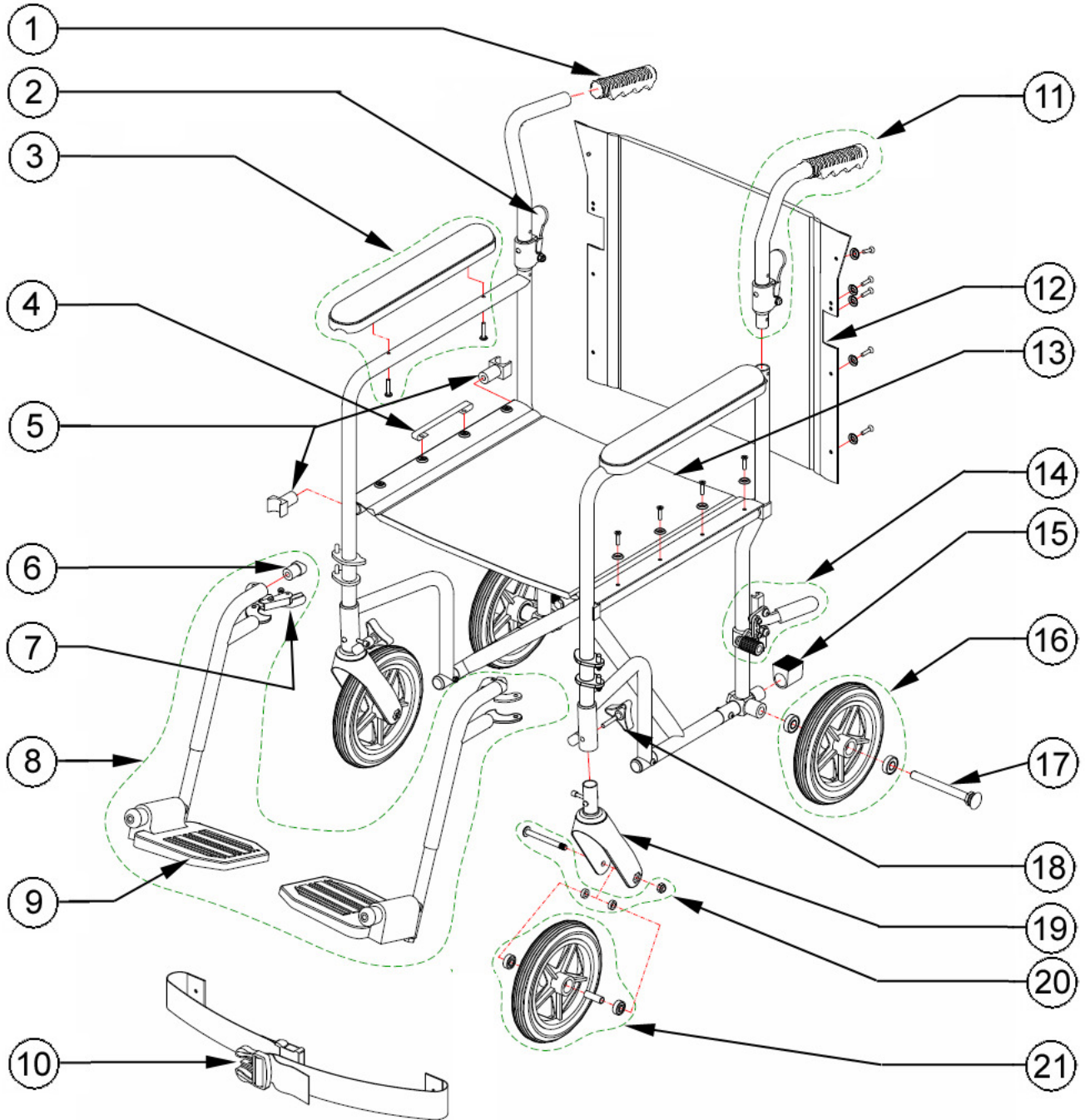

**700-869 (MIDNIGHT-BLUE)  
HUGO TRANSPORT CHAIR 19" WIDE**

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	<b>A07-401</b>	Hand Grip, Black (2 ea/set)	1	1	ST
2.	<b>A07-421</b>	Release Lever to Fold Handle, Black (2 ea/set)	1	1	ST
3.	<b>B07-251</b>	Full Armrest, Black Nylon with Hardware (2 Arm Pads 360x57mm, 4 Round Philips Screws M5x25mm)	1	1	ST
4.	<b>A07-601</b>	Seat Strap, Black Nylon (2 ea /set)	1	1	ST
5.	<b>A07-411</b>	Seat Guide (Front & Back), Black (4 ea/set)	1	1	ST
6.	<b>A07-431</b>	Bumper for Footrest, Black (2 ea/set)	1	1	ST
7.	<b>B07-391</b>	Release Handle Mechanism with Hardware, Left & Right (2 Long Handle L+ R, 4 Philips Bolts: 2 M5x20mm + 2 M5x25mm, 2 Hex Locknuts M5, 6 Spacers: 2 Long + 2 Short + 2 Special and 2 Springs)	1	1	PR
8.	<b>B02-351</b>	Detachable Swing-Away Footrests, Blue	1	1	PR
9.	<b>A07-451</b>	Foot Plate, Black (2 ea/set)	1	1	ST
10.	<b>700-864</b>	Safety Belt, Black	1	1	EA
11.	<b>B07-303</b>	Folding Blue Handles (Left & Right Blue Handle Bar, Hand Grips & Folding Mechanism)	1	1	PR
12.	<b>B07-402</b>	Hugo Logo Back Upholstery 19"x16", Black Nylon, with Hardware (2 Supporting Plates 13x195mm, 10 Self-threading Screws M5x17mm & 10 Black Saddle Washers)	1	1	EA
13.	<b>B02-431</b>	Seat Upholstery 19"x15.5", Black Nylon, with Hardware	1	1	EA
14.	<b>B02-211</b>	Wheel Lock Assembly for Transport Chair, Left & Right	1	1	PR
15.	<b>A07-435</b>	Plug for Rear Frame, Black	1	1	ST
16.	<b>B07-126</b>	8" Solid Rear Wheel Assembly, Black (Black Tire Wheel with 8 Spokes, 2 Bearings 6001Z & Bushing)	2	1	EA
17.	<b>B07-861</b>	Quick-Release Bolt System for Wheel	2	1	EA
18.	<b>B07-831</b>	Front Fork Secure Knob (Triangular Knob, Hex Bolt M6x35mm & Curved Washer)	2	1	ST
19.	<b>B07-151</b>	Front Fork Assembly (Plastic Fork with 2 Holes, 2 Bearings 6001, Bushing, Locknut M12, Plastic Cover, Stem M12, Push-Buttons)	2	1	EA
20.	<b>B07-851</b>	Hardware for 8" Front Wheel (Hex Socket Bolt M8x78mm, 2 Spacers & Hex Locknut M8)	2	1	ST
21.	<b>B07-101</b>	8" Solid Front Wheel Assembly, Black (Black Tire Wheel with 8 Spokes, 2 Bearings 608 & Bushing)	2	1	EA

**700-869 (MIDNIGHT-BLUE)  
HUGO ULTRA-LIGHT WEIGHT, KD TRANSPORT CHAIR 19" WIDE**

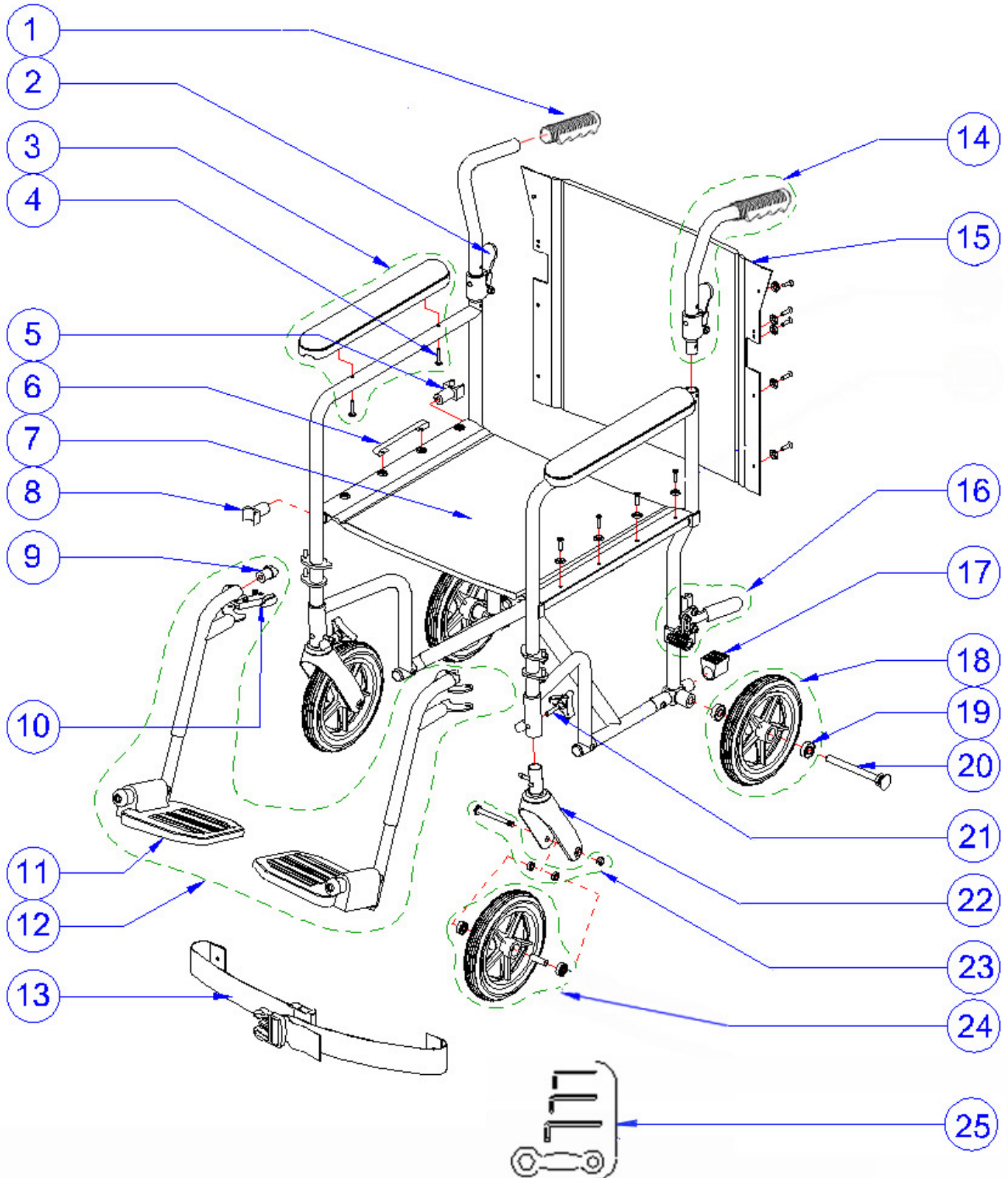

**700-869 (MIDNIGHT-BLUE)  
HUGO ULTRA-LIGHT WEIGHT, KD TRANSPORT CHAIR 19" WIDE**

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	<b>A02-401</b>	Ortho-Hand Grip, Black (L&R)	1	1	PR
2.	<b>A02-411</b>	Release Lever to Fold Handle, Black (2 ea/set)	1	1	ST
3.	<b>A02-461</b>	Full Arm Pad, Black Nylon	1	1	PR
4.	<b>A02-122</b>	Hex Socket Screw (M5x30mm) – 4 ea/set	1	1	ST
5.	<b>A02-485</b>	Rear Seat Guide, Black Plastic	2	1	EA
6.	<b>A02-476</b>	Seat Strap, Black Nylon (2 ea /set)	1	1	ST
7.	<b>B02-431</b>	Hugo Seat Upholstery 19"x15.5", Black Nylon, with Hardware (8 Hex Screws & 8 Washers)	1	1	EA
8.	<b>A02-484</b>	Front Seat Guide, Black Plastic	2	1	EA
9.	<b>A02-430</b>	Bumper for Footrest, Black	2	1	EA
10.	<b>B02-341</b>	Release Mechanism for Footrest (Handle, 2 Bolts, 2 Hex Nuts & Spring), Left & Right	1	1	PR
11.	<b>A02-458</b>	Foot Plate, Black	2	1	EA
12.	<b>B02-351</b>	Swing-Away Footrests, Midnight-Blue, Left & Right	1	1	PR
13.	<b>700-864</b>	Safety Belt for Transport Chair	1	1	EA
14.	<b>B02-081</b>	Folding Handle Bar with Release Mechanism, Midnight-Blue (L&R)	1	1	PR
15.	<b>B02-465</b>	Hugo Back Upholstery 19"x17.5", Black Nylon, with Hardware (10 Self-threading Screws M5x17mm & 10 Black Saddle Washers)	1	1	EA
16.	<b>B02-211</b>	Wheel Lock Assembly for Light Transport Chair (L&R)	1	1	PR
17.	<b>A02-421</b>	Step Tube Plug, Black	2	1	EA
18.	<b>B02-112</b>	8"x1 1/4" PU Rear Wheel, Black (Black Tire Wheel with 5 Spokes, 2 Bearings 608Z & 1 Bushing)	2	1	EA
19.	<b>A02-301</b>	608 Bearing (ø22xø8x8mm)	8	1	EA
20.	<b>B02-838</b>	Quick-Release Bolt System for Wheel	2	1	EA
21.	<b>A02-041</b>	Front Fork Secure Triangular Knob (Hex Bolt M6x35mm)	2	1	EA
22.	<b>B02-147</b>	Plastic Fork Assembly for 8" Wheel (Plastic Fork with 2 Holes, 2 Bearings 6001, Bushing, Locknut M12, Plastic Cover, Stem M12, Push-Buttons)	2	1	EA
23.	<b>B02-837</b>	Hardware for 8" Front Wheel (Hex Socket Bolt M8x75mm, 2 Spacers & Locknut M8)	2	1	ST
24.	<b>B02-111</b>	8" Solid Front Wheel Assembly, Black (Black Tire Wheel with 5 Spokes, 2 Bearings 608 & Bushing)	2	1	EA
25.	<b>A02-703</b>	Tool Package (4, 5 & 6 mm Allen Keys and Wrench for 13 & 19 mm)	1	1	ST