


**793-943 (PACIFIC-BLUE)**  
**HUGO NAVIGATOR ROLLATOR & TRANSPORT CHAIR**  
**SIDE-FOLDING, HUGO HANDLES AND COMFORT-GRIP WHEELS**

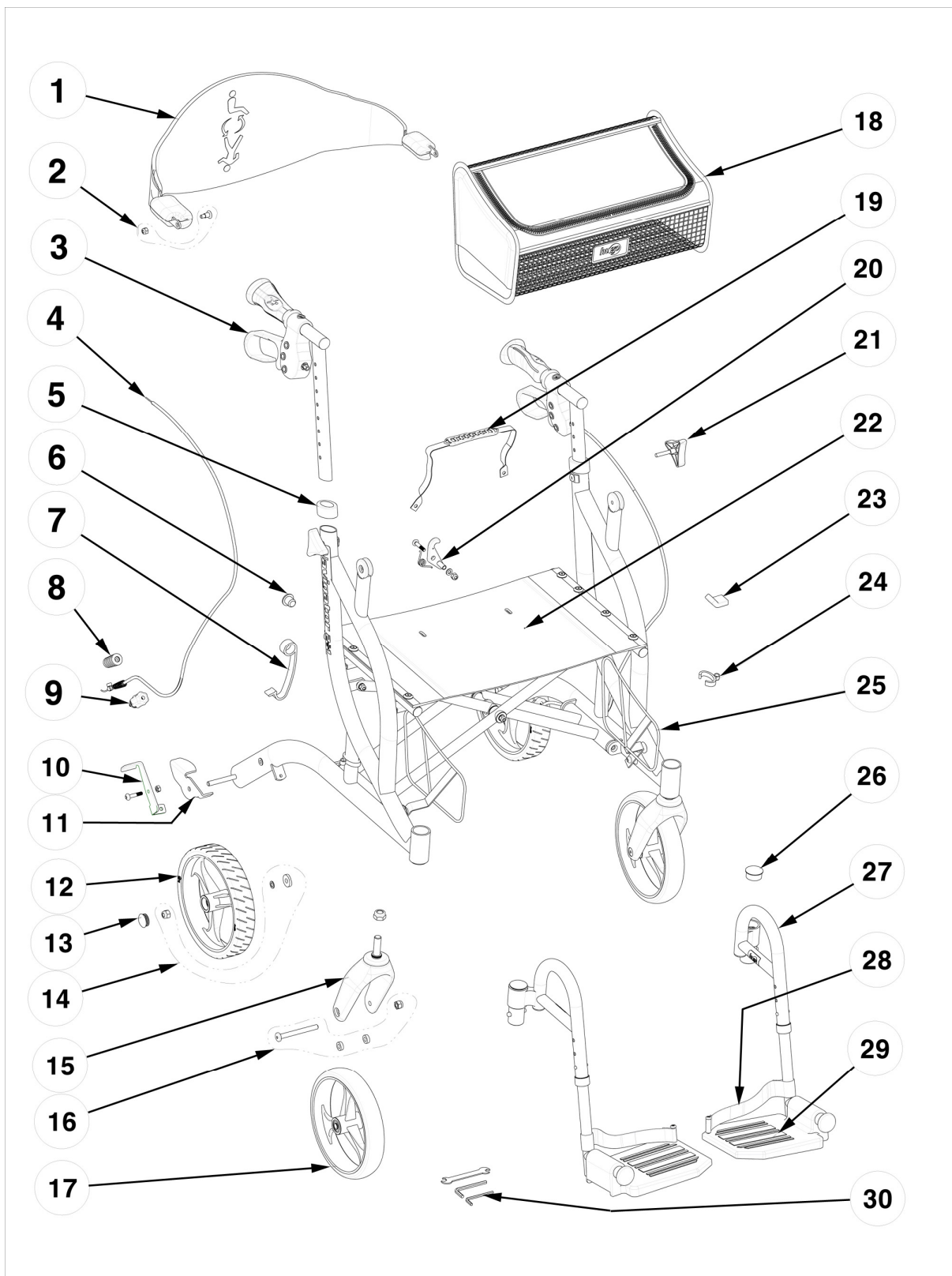

**793-943 (PACIFIC-BLUE)**  
**HUGO NAVIGATOR ROLLATOR & TRANSPORT CHAIR**  
**SIDE-FOLDING, HUGO HANDLES AND COMFORT-GRIP WHEELS**

| Item # | AMG Part #       | Item Description                                                                                                                                        | Qty Total | Qty Sold |
|--------|------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------|-----------|----------|
| 1.     | <b>C23-00122</b> | Full Contoured Backrest with Armrests, Black                                                                                                            | 1 EA      | 1 EA     |
| 2.     | <b>B01-861</b>   | Hardware for Backrest (2 Special Bolts M6x16mm & 2 Hex Locknuts)                                                                                        | 1 ST      | 1 ST     |
| 3.     | <b>C23-00123</b> | Blue-Grips Handle Oval Bar with Brake Handle Subassy                                                                                                    | 1 PR      | 1 PR     |
| 4.     | <b>B01-181</b>   | Brake Cable & Sleeve Assembly (Cable, Sleeve 860mm, Bottom Brake Adjustment, Spring, Spring Sleeve, Cable Stopper with Screw M5x6mm & Fishtail Stopper) | 1 PR      | 1 PR     |
| 5.     | <b>A01-211</b>   | Oval Anti-Rattle, Black Plastic – 2ea / set                                                                                                             | 1 ST      | 1 ST     |
| 6.     | <b>B01-601</b>   | Caps for Seat Tubing, Black (2 Front Caps & 2 Rear Caps)                                                                                                | 1 ST      | 1 ST     |
| 7.     | <b>A01-282</b>   | Fold Stopper Strap, Orange Plastic                                                                                                                      | 1 EA      | 1 EA     |
| 8.     | <b>A01-419</b>   | Spring Sleeve, Black                                                                                                                                    | 2 EA      | 1 EA     |
| 9.     | <b>B01-185</b>   | Bottom Brake Adjustment Assy (Orange Knob, Spring & Orange Adj. Screw)                                                                                  | 2 ST      | 2 ST     |
| 10.    | <b>C23-00027</b> | Bottom Brake Assy, L&R (2 Bolts M6x22mm, Left & Right Brake Pad & 2 Hex Nuts)                                                                           | 1 ST      | 1 ST     |
| 11.    | <b>C23-00117</b> | Curb Climber, right side                                                                                                                                | 1 EA      | 1 EA     |
| 12.    | <b>B01-147X</b>  | 8"x1.5" Hugo Rear Wide Wheel                                                                                                                            | 2 EA      | 1 EA     |
| 13.    | <b>A01-453</b>   | Hub Cap for Rear Wheel                                                                                                                                  | 2 EA      | 1 EA     |
| 14.    | <b>B01-845</b>   | Hardware for Rear Wheel (Locknut M8, Curved Washer & Steel Washer)                                                                                      | 2 ST      | 1 ST     |
| 15.    | <b>B01-303</b>   | Front Fork Subassy,(Black Plastic Fork, 2 Bearings, Fork Cover, Fork Bolt M12x65mm, Aluminum Spacer, Spring-Washer & Locknut M12)                       | 2 EA      | 1 EA     |
| 16.    | <b>B01-832</b>   | Hardware for 8" Front Wheel (Bolt M8x78.5mm, 2 Aluminum Spacers & Hex Nut M8)                                                                           | 2 ST      | 1 ST     |
| 17.    | <b>B01-146</b>   | 8"x1.4" Hugo Front Wheel                                                                                                                                | 2 EA      | 1 EA     |
| 18.    | <b>C23-00124</b> | Soft Basket for Hugo, closed                                                                                                                            | 1 EA      | 1 EA     |
| 19.    | <b>A01-286</b>   | Seat Strap, Black                                                                                                                                       | 1 EA      | 1 EA     |
| 20.    | <b>B01-881W</b>  | Open Seat Mechanism (Right Link Bar, Bolt M6x28mm, Alum. Bracket, Spring, Chromed Ring, Brass Sleeve, 2 Grey Plastic Washers & Hex Nut M6)              | 1 ST      | 1 ST     |
| 21.    | <b>A01-416</b>   | Triangular Knob, M6x30mm – 2ea / set                                                                                                                    | 1 ST      | 1 ST     |
| 22.    | <b>B01-282</b>   | Vinyl Foldable Seat XWD (19.5" x 12") with Hardware (8 screws M4x14mm & 8 Black Plastic Washers)                                                        | 1 EA      | 1 EA     |
| 23.    | <b>A01-450</b>   | Seat Support Bracket Cover, Black - 2 ea / set                                                                                                          | 2 ST      | 1 ST     |
| 24.    | <b>A01-244</b>   | Oval Cable-Clip, Black – 2 ea / set                                                                                                                     | 1 ST      | 1 ST     |
| 25.    | <b>B01-873</b>   | Mounting Brackets with Hardware, Fusion                                                                                                                 | 1 ST      | 1 ST     |
| 26.    | <b>A01-455</b>   | Cap for Front 34mm Tubing, Black – 2ea / set                                                                                                            | 1 ST      | 1 ST     |
| 27.    | <b>C23-00125</b> | Footrest Assembly – L & R                                                                                                                               | 1 PR      | 1 PR     |
| 28.    | <b>A01-291</b>   | Footrest Strap for Fusion / Navigator - 2 ea / set                                                                                                      | 1 ST      | 1 ST     |
| 29.    | <b>B01-641</b>   | Footplate for Footrest - L & R                                                                                                                          | 1 PR      | 1 PR     |
| 30.    | <b>A01-712</b>   | Tool Package of 3 ( #4mm, #5mm Allen Keys & 8-10mm Wrench)                                                                                              | 1 ST      | 1 ST     |