


SENTRA RECLINER

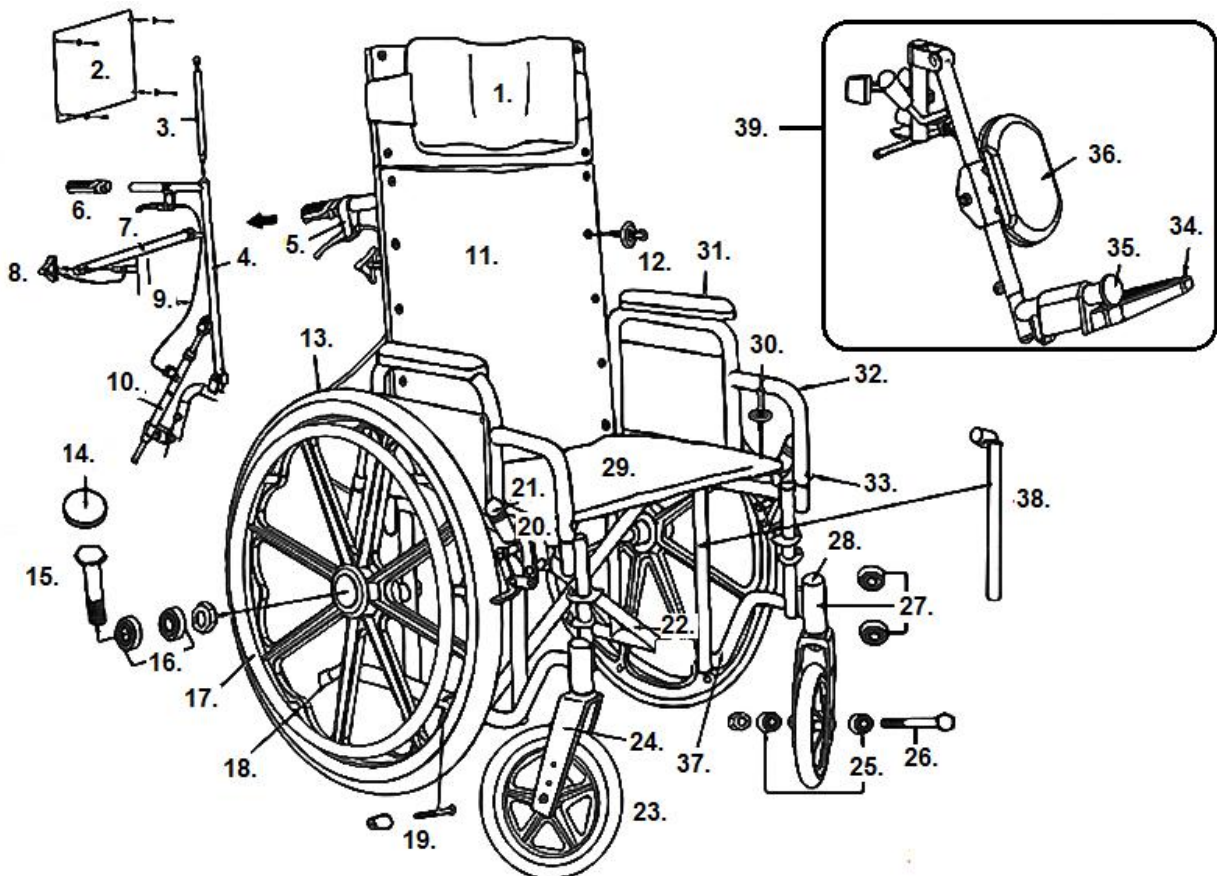

- | | |
|---|--|
| <p>1. Headrest Pillow 14" to 20"- STDS2010
Headrest Pillow 22"- STDS2010-22</p> <p>2. Headrest Assembly 14" *- STDS2A2914
Headrest Assembly 16" *- STDS2012
Headrest Assembly 18" *- STDS2011
Headrest Assembly 20" *- STDS2A2920
Headrest Assembly 22" *- STDS2A2922
Headrest Uphol. Only 14"- STDS2A2904
Headrest Uphol. Only 16"- STDS2A2906
Headrest Uphol. Only 18"- STDS2A2908
Headrest Uphol. Only 20"- STDS2A2910
Headrest Uphol. Only 22"- STDS2A2912</p> <p>3. Headrest Post- STDS2550</p> <p>4. Backrest Cane- STDS905</p> <p>5. Recline Release Lever Left **- STDS4041
Recline Release Lever Right **- STDS4040</p> <p>6. Hand Grip- STDS871</p> <p>7. Spreader Bar 14"- STDS2A2924
Spreader Bar 16"- STDS4S4809
Spreader Bar 18"- STDS4S4810
Spreader Bar 20"- STDS4S4811
Spreader Bar 22"- STDS2A2932</p> <p>8. Knob- STDS900K</p> <p>9. Cable- STDS3006</p> <p>10. Reclining Piston Left- STDS4014L
Reclining Piston Right- STDS4014R</p> <p>11. Back Upholstery 14"- STDS2A2944
Back Upholstery 16"- STDS2295BLK
Back Upholstery 18"- STDS2035BLK
Back Upholstery 20"- STDS2327BLK
Back Upholstery 22"- STDS2A2923</p> <p>12. Back Upholstery Hardware (8 ea)- STDS5D5504</p> <p>13. Rear Wheel/Black Hand Rim- STDS1006
Rear Wheel/Chrome Hand Rim- STDS1000
Rear Wheel/Aluminum Hand Rim- STDS1001
Rear Wheel 22" Recliner- STDS1000HD</p> <p>14. Hub Cap- STDS1000C</p> <p>15. Rear Axle- STDS1005-1
Rear Axle 22" Recliner- STDS1052</p> <p>16. Rear Wheel Bearing- STDS1012
Rear Wheel Bearing 22" Recliner- STDS1012HD</p> <p>17. Chrome Hand Rim- STDS004010
Aluminum Hand Rim- STDS004011
Black Hand Rim- STDS004012</p> <p>18. Anti-Tippers(pr)- STDS818</p> <p>19. Hdwr. Hand Rim with Rivnuts (8ea)- STDS1073
Hdwr. Hand Rim w/o Rivnuts (ea)- STDS000021</p> | <p>20. Wheel Lock Left- STDS4035L
Wheel Lock Right- STDS4035R</p> <p>21. Wheel Lock Tip- STDS4011</p> <p>22. Cross Frame Bolt (14" to 20")- STDS001061</p> <p>23. Caster 8" (14" to 20")- STDS1002
Caster 6" (14" to 20")- STDS1074
Caster 8" (22")- STDS1075</p> <p>24. Fork (14" to 20")- STDS3003
Fork (22")- STDS3045-2
Fork 6" Caster- STDS3045-2</p> <p>25. Caster Bearing (6" & 8")- STDS1011</p> <p>26. Caster Axle (14" to 20")- STDS1002-1
Caster Axle (22")- STDS1014</p> <p>27. Fork Stem Bearing (14" to 20")- STDS1060
Fork Stem Bearing (22")- STDS8M4601</p> <p>28. Dust Cap- STDS1051</p> <p>29. Seat Upholstery 14"- STDS2A2934
Seat Upholstery 16"- STDS2324BLK
Seat Upholstery 18"- STDS2036BLK
Seat Upholstery 20"- STDS2047BLK
Seat Upholstery 22"- STDS2A2942</p> <p>30. Seat Upholstery Hardware (8 ea)- STDS1072-8</p> <p>31. Desk Arm Pad- STDS4011
Full Arm Pad- STDS4012</p> <p>32. Arm Rest Assembly Desk Left- STDS4013L
Arm Rest Assembly Desk Right- STDS4013R
Adj. Ht. Arm Rest Assy. Desk Left- STDS4014L
Adj. Ht. Arm Rest Assy. Desk Rt.- STDS4014R
Arm Rest Assembly Full Left- STDS4015L
Arm Rest Assembly Full Right- STDS4015R
Adj. Ht. Arm Rest Assy. Full Left- STDS4016L
Adj. Ht. Arm Rest Assy. Full Rt.- STDS4016R</p> <p>33. Arm Release Button- STDS4017</p> <p>34. Foot Plate (14")- STDS003441
Foot Plate (16" to 22")- STDS00004A</p> <p>35. Footrest Bumper- STDS1026</p> <p>36. Calf Pad (14")- STDS2A3442
Calf Pad (16" to 22")- STDS4Q2007BLK</p> <p>37. Side Frame Left (14" to 20") - STDS2A291L
Side Frame Right (14" to 20") - STDS2A291R
Side Frame Left (22") - STDS2A292L
Side Frame Right (22") - STDS2A292R</p> <p>38. Slide Tube- STDS2A2953</p> <p>39. Elevating Leg Rest (14")- JLELR-TF
Elevating Leg Rest (16" to 22")- STDELRTF</p> |
|---|--|

* Does not include pillow.

** Does not include cable.