


SILVER SPORT 1 (1Y, 3J & 5D SERIAL #)

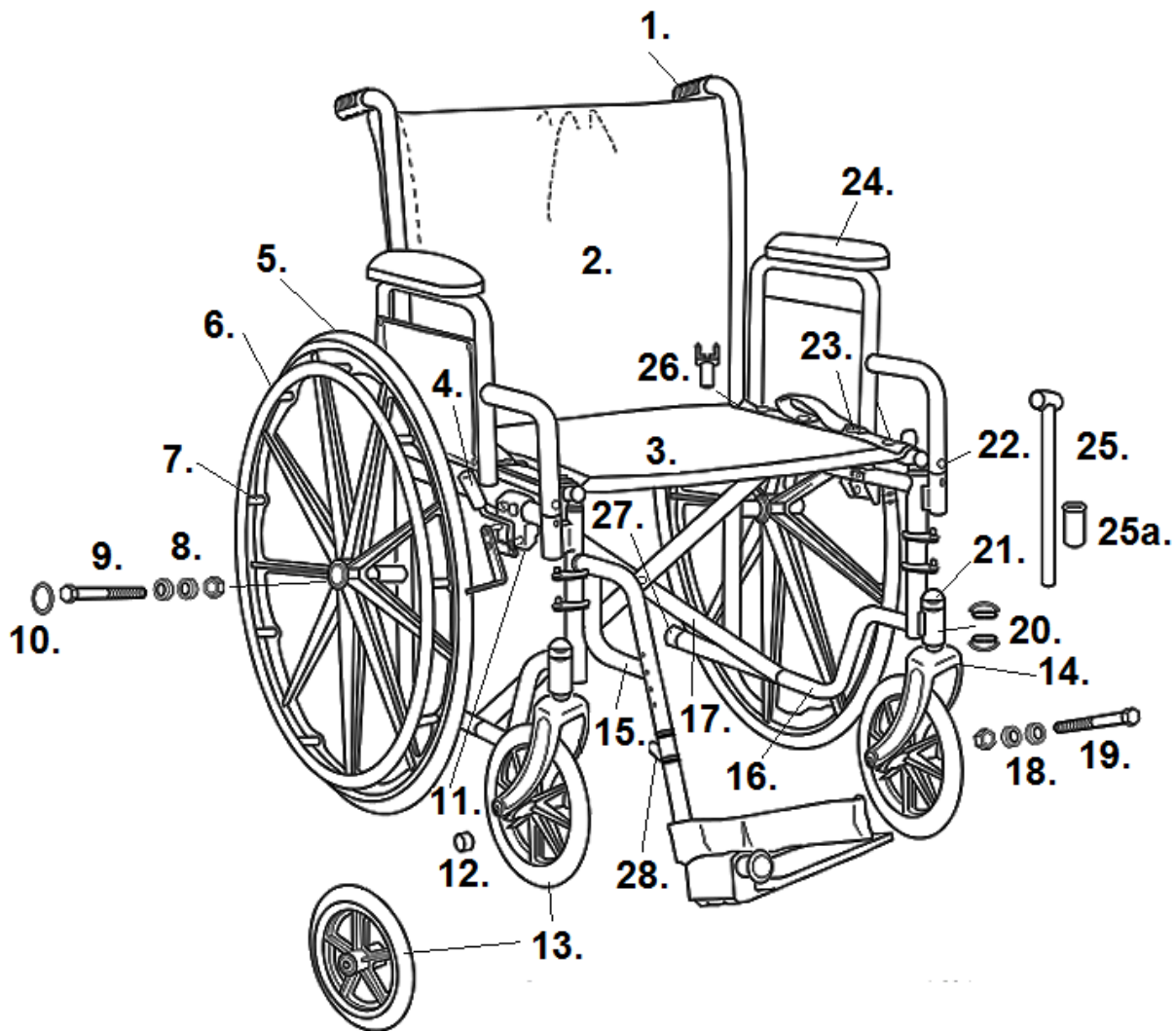

- | | |
|---|---|
| 1. Hand Grip- WB 8036 | 16. Side Frame Left- STDS3J45L1
Side Frame Right- STDS3J45R1 |
| 2. Back Uphol. 18" 1Y & 3J Serial #- STDS3J4508
Back Upholstery 18" 5D Serial #- STDS5D4508 | 17. Cross Frame 18"- STDS3J45X8 |
| 3. Seat Uphol. 18" 1Y & 3J Serial #- STDS3J2418
Seat Upholstery 18" 5D Serial #- STDS3J2418 | 18. Caster Bearing (8 mm)- STDS1011 |
| 4. Wheel Lock Tip- STDS8M4603 | 19. Caster Axle Bolt- STDS3J2431 |
| 5. Rear Wheel- STDS3J2424 | 20. Fork Stem Bearing (12.5 mm)- STDS8M4601
Fork Stem Bearing Flanged (13 mm)- STDS3J2401 |
| 6. Hand Rim- STDS3J2400 | 21. Fork Stem Cap- STDS8M4651 |
| 7. Hand Rim Hardware (9 ea)- STDS004609-9 | 22. Arm Release Button- STDS3J4503 |
| 8. Rear Wheel Bearing (11 mm)- STDS3J2413 | 23. Upholstery Hardware Back/Seat (8 ea)- STDS2015 |
| 9. Rear Axle Bolt Detachable Arms- STDS3J4626
Rear Axle Bolt Fixed Arms- STDS3J4625 | 24. Desk Arm Pad- STDS8M4651
Full Arm Pad- STDS8M4651 |
| 10. Hub Cap- STDS3J2421 | Desk Arm Left- STDS8M4651
Desk Arm Right- STDS8M4651 |
| 11. Wheel Lock Fixed Arms Left- STDS8M241L
Wheel Lock Fixed Arms Right- STDS8M241R
Wheel Lock Detachable Arms Lf- STDS8M462L
Wheel Lock Detachable Arms Rt- STDS8M462R | Full Arm Left- STDS8M4651
Full Arm Right- STDS8M4651 |
| 12. Axle Cap- STDS001000 | 25. Slide Tube & Bushing- STDS3J4601 |
| 13. Caster 8 Spoke- STDS3J2432
Caster 5 Spoke- STDS3J3900 | 25a. Bushing- STDS002601 |
| 14. Fork- STDS3J4533 | 26. Seat Rail Guide Detachable Arms- STDS1070
Seat Rail Guide Fixed Arms Front- STDS004602
Seat Rail Guide Rear- STDS1070 |
| 15. Footrest (pr)- STDS3J24SF
Elevating Leg Rest (pr)- LK3JELR | 27. Tipping Sleeve- STDS3J2422 |
| | 28. Footrest Adj. Lever- STDS3J4502 |