


SILVER SPORT 1 (4S & 8M SERIAL #)

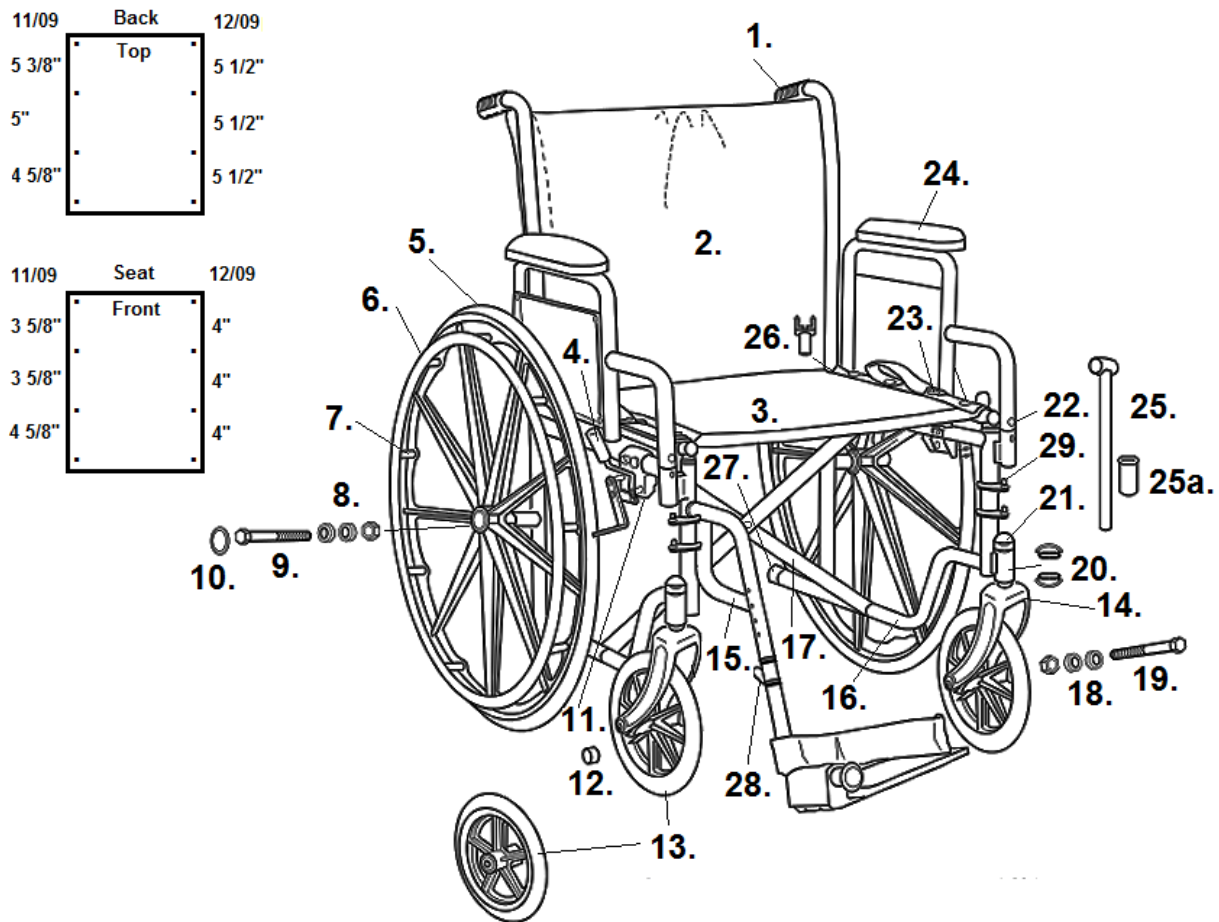

- 1. Hand Grip- WB 8036
- 2. Back Uphol. 18" 11/09 & Before- STDS8M4508
Back Uphol. 18" 12/09 & After- SRDS3J4508
- 3. Seat Uphol. 18" 11/09 & Before- STDS8M4518
Seat Uphol. 18" 12/09 & After- STDS3J2418
- 4. Wheel Lock Tip- STDS8M4603
- 5. Rear Wheel- STDS3J2424
- 6. Hand Rim- STDS3J2400
- 7. Hand Rim Hardware (9 ea)- STDS004609-9
- 8. Rear Wheel Bearing (11 mm ID)- STDS3J2413
- 9. Rear Axle Bolt Detachable Arms- STDS3J4626
- 10. Hub Cap- STDS3J2421
- 11. Wheel Lock Fixed Arms Left- STDS8M241L
Wheel Lock Fixed Arms Right- STDS8M241R
Wheel Lock Detachable Arms Left- STDS8M462L
Wheel Lock Detachable Arms Right- STDS8M462R
- 12. Axle Cap- STDS001000
- 13. Caster 8 Spoke- STDS3J2432
- 14. Fork- STDS3J4533
- 15. Footrest (pr)- STDS3J24SF
Elevating Leg Rest (pr)- LK3JELR
- 16. Side Frame Left- STDS3J45L1
Side Frame Right- STDS3J45R1
- 17. Cross Frame 18" 11/09 & Before *
- Cross Frame 18" 12/09 & After- STDS3J45X8
- 18. Caster Bearing (8 mm)- STDS1011
- 19. Caster Axle Bolt- STDS3J2431
- 20. Fork Stem Bearing (12.5 mm)- STDS8M4601
- 21. Fork Stem Cap- STDS8M4651
- 22. Arm Release Button- STDS3J4503
- 23. Upholstery Hardware Back/Seat (8 ea)- STDS2015
- 24. Desk Arm Pad- STDS3J24VB
Full Arm Pad- STDS3J24VFB
Desk Arm Left- STDS3J24VLDL
Desk Arm Right- STDS3J24VDR
Full Arm Left- STDS3J24VFL
Full Arm Right- STDS3J24VFR
- 25. Slide Tube & Bushing- STDS3J4601
- 25a. Bushing- STDS002601
- 26. Seat Rail Guide Detachable Arms- STDS1070
Seat Rail Guide Fixed Arms Front- STDS004602
Seat Rail Guide Rear- STDS1070
- 27. Tipping Sleeve- STDS3J2422
- 28. Footrest Adj. Lever- STDS3J4502
- 29. Hanger Pins 4S Serial # (4 ea)- STDS4S1069

* No Longer Available