


VIPER PLUS GT

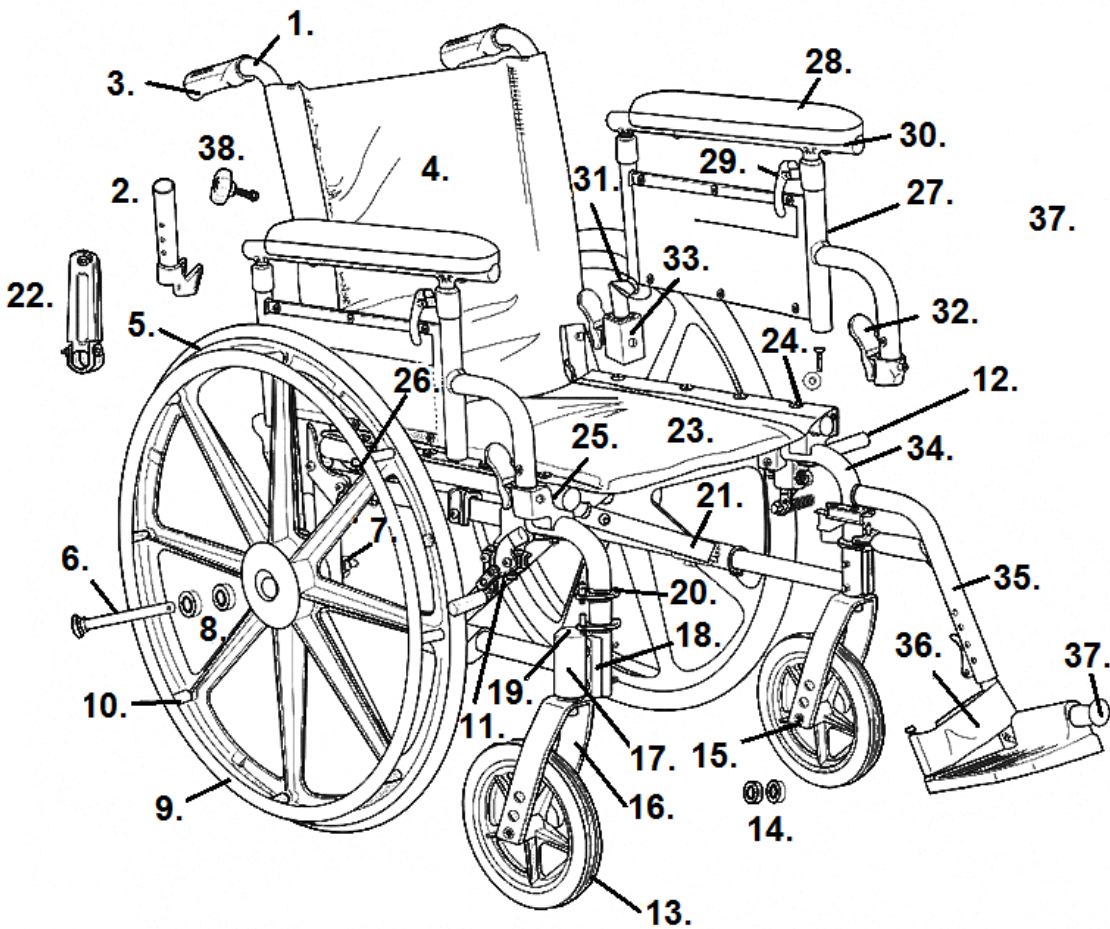

- | | |
|--|--|
| <p>1. Backrest Cane (pr)- STDS1A3557
 2. Backrest Pivot Bracket- STDS2A3250
 3. Hand Grip- STDS871
 4. Back Upholstery 14"- *
 Back Upholstery 16"- STDS2363BLK
 Back Upholstery 17"- *
 Back Upholstery 18"- STDS2359BLK
 Back Upholstery 19"- *
 Back Upholstery 20"- STDS2361BLK
 Back Upholstery 22"- STDS2369BLK
 5. Rear Wheel- STDS1A3510
 6. Quick Release Axle- STDS1A3511
 7. Rear Axle receiver- STDS1A3512
 8. Rear Wheel Bearing- STDS5D1012
 9. Hand Rim- STDS2A3511
 10. Hand Rim Hardware (8 ea)- STDS1073
 11. Push To Lock Wheel Lock Left- STDS1A351L
 Push To Lock Wheel Lock Right- STDS1A351R
 Pull To Lock Wheel Lock Left- STDS1A352L
 Pull To Lock Wheel Lock Right- STDS1A352R
 Wheel Lock Bracket- STDS1A351-BC
 12. Wheel Lock Tip- STDS2A4335
 13. Caster- STDS1A3500
 14. Caster Bearing- STDS1011
 15. Caster Axle- STDS1A3513
 16. Fork- STDS1A3503
 17. Fork Stem Bearing- STDS1060
 18. Fork Bearing Housing Left- STDS1A350L
 Fork Bearing Housing Right- STDS1A350R
 19. Dust Cap- STDS1051
 20. Hanger Pins (4 ea)- STDS1069-4
 21. Cross Frame Non-Adjustable 14"- *
 Cross Frame Non-Adjustable 16"- STDS2A35X6
 Cross Frame Non-Adjustable 17"- *
 Cross Frame Non-Adjustable 18"- STDS2A35X8
 Cross Frame Non-Adjustable 19"- *
 Cross Frame Non-Adjustable 20"- STDS2A35X0
 Cross Frame Non-Adjustable 22"- STDS2A35X2
 Cross Frame Adjustable 16"- STDS2A35X6-AD
 Cross Frame Adjustable 18"- STDS2A35X8-AD
 Cross Frame Adjustable 20"- STDS2A35X0-AD
 Adj. Depth Extension Tube- STDS2A35X-ET</p> | <p>22. Cross Frame Assist 14"- *
 Cross Frame Assist 16"-18"-20"- STDS1A3501N
 Cross Frame Assist 19"- *
 Cross Frame Assist 22"- STDS1A3502
 23. Seat Upholstery Non-Adjustable 14"- *
 Seat Upholstery Non-Adj. 16"- STDS2356BLK
 Seat Upholstery Non-Adjustable 17"- *
 Seat Upholstery Non-Adj. 18"- STDS2364BLK
 Seat Upholstery Non-Adjustable 19"- *
 Seat Upholstery Non-Adj. 20"- STDS2366BLK
 Seat Upholstery Non-Adj. 22"- STDS2368BLK
 Seat Upholstery Adj. 16"- STDS2356BLK-AD
 Seat Upholstery Adj. 18"- STDS2364BLK-AD
 Seat Upholstery Adj. 20"- STDS2366BLK-AD
 24. Seat Upholstery Hardware (8 ea)- STDS1072-8
 25. Front Seat Saddle Left- STDS2A3140
 Front Seat Saddle Right- STDS2A3141
 26. Rear Seat Saddle- STDS4058
 27. Adj. Height Armrest Desk Left- STDS1A35DL
 Adj. Height Armrest Desk Right- STDS1A35DR
 Adj. Height Armrest Full Left- STDS1A35FL
 Adj. Height Armrest Full Right- STDS1A35FR
 28. Arm Pad Desk- STDS2SKNB
 Arm Pad Full- STDS2FLNB
 29. Adjustment Lever Left- STDS2A0044
 Adjustment Lever Right- STDS2A0046
 30. Upper Arm & Pad Desk Left- STDS1A35UDL
 Upper Arm & Pad Desk Right- STDS1A35UDR
 Upper Arm & Pad Full Left- STDS1A35UFL
 Upper Arm & Pad Full Right- STDS1A35UFR
 31. Pivot Bracket- STDS2A3240
 32. Arm Release Lever- STDS2A3144
 33. Rear Arm Bracket Upper- STDS1A3541
 Rear Arm Bracket Lower- STDS1A3542
 34. Side Frame Left- STDS2A353L
 Side Frame Right- STDS2A353R
 35. Footrest- AL3-4SF
 Elevating Leg Rest- ALELR
 36. Heel Loop- STDS831
 37. Bumper- STDS1026
 38. Back Height Adjustment Knob- STDS895
 * NO LONGER AVAILABLE
 (Not Shown) Anti-Tippers- STDS807</p> |
|--|--|