


Nitro Tall

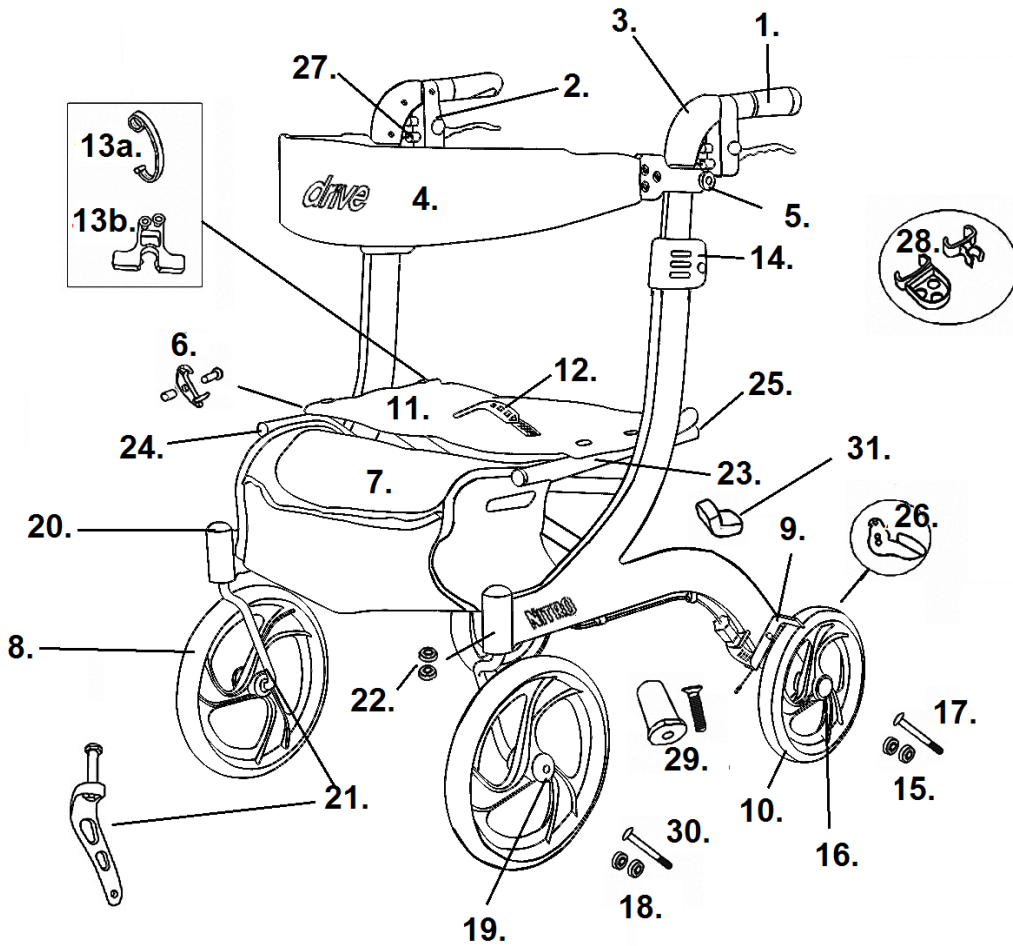

- | | |
|---|--|
| 1. Handgrip Left- 1026602
Handgrip Right- 1026603 | 12. Folding Strap Assy. 10266-T- 1026609-FSR-TALL
Folding Strap Assy. 10266BK-T- 1026609-FSG-TALL
Folding Strap Assy. 10266WT-T- 1026609-FSW-TALL |
| 2. Hand Brake RTL10266-T Lt- 1026601-RN-T
Hand Brake RTL10266-T Rt- 1026601-RNR-T
Hand Brake RTL10266BK-T Lt- 1026601-G-T
Hand Brake RTL10266BK-T Rt- 1026601-GR-T
Hand Brake RTL10266WT-T Lt- 1026601-W-T
Hand Brake RTL10266WT-T Rt- 1026601-WR-T
Cable Only 55.5"- 1026632-T | 13a. Folding Locking Latch 10266-T- 1026626-R
Folding Locking Latch 10266BK-T- 1026626-G
Folding Locking Latch 10266WT-T- 1026626-W |
| 3. Handle Bar- 1026630N | 13b. Folding Locking Latch- 1026644 [3] |
| 4. Backrest Assembly RTL10266-T- 1026604-B
Backrest Assembly RTL10266WT-T- 1026604-B
Backrest Assembly RTL10266BK-T- 1026604-G | 14. Back Height Adj. Latch RTL10266-T- 1026607-R
Back Height Adj. Latch RTL10266BK-T- 1026607-G
Back Height Adj. Latch RTL10266WT-T- 1026606-W |
| 5. Backrest Knob & Bolt RTL10266-T- 1026605-R
Backrest Knob & Bolt RTL10266WT-T- 1026605-W
Backrest Knob & Bolt RTL10266BK-T- 1026605-G | 15. Rear Axle Bearings (pr)- 1026613 |
| 6. Safety Lock- 1026638 | 16. Rear Hub Cap- 1026619 |
| 7. Tote Bag RTL10266-T- 1026633-R
Tote Bag RTL10266BK-T- 1026633-BK
Tote Bag RTL10266WT-T- 1026633-WT | 17. Rear Axle- 1026631 |
| 8. Caster & Fork Assembly Style #1 Left- 1026625
Caster & Fork Assembly Style #1 Right- 1026624
Caster & Fork Assembly Style #2 Left- 1026625-AN [4]
Caster & Fork Assy. Style #2 Right- 1026624-AN [4]
Caster Style #1- 1026617
Caster Style #2- 1026617-AN [4] | 18. Front Axle Bearings (pr)- 1026618
Front Axle Bearings (pr)- 1026618-AN [4] |
| 9. Wheel lock Assembly Metal Left- 1026615
Wheel lock Assembly Metal Right- 1026616
Wheel lock Assembly Plastic Left- 1026641 [2]
Wheel lock Assembly Plastic Right- 1026642 [2] | 19. Front Hub Cap Old Style Fork- 1026614
Front Hub Cap New Style Fork- 1026614-AN [4] |
| 10. Rear Wheel Left- 1026612-L
Rear Wheel Right- 1026612-R | 20. Dust Cap (pr)- 1026620 |
| 11. Seat/Hdwr. RTL10266-T (Black)- 1026608-TALL
Seat/Hdwr. RTL10266WT-T (Grey)- 1026608-TALL-G
Seat/Hdwr. RTL10266BK-T (Grey)- 1026608-TALL-G | 21. Fork Assembly Old Style Left- 1026622
Fork Assembly Old Style Right- 1026621
Fork Assembly New Style Left- 1026622-AN [4]
Fork Assembly New Style Right- 1026621-AN [4] |
| | 22. Fork Stem Bearings (pr)- 1026623 |
| | 23. Cross Frame- 1026627-TALL
Cross Frame Bolt- 1026646 |
| | 24. Cross Frame Front End Cap (ea)- 1026634 |
| | 25. Cross Frame Rear End Caps (pr)- 1026628 |
| | 26. Step Tube Left- 1026636 [1]
Step Tube Right- 1026637 [1] |
| | 27. Brake Cable Guide- 1026639 |
| | 28. Cane Holder- 892 |
| | 29. Cross Brace Screw & Insert- 1026643 |
| | 30. Front Axle- 1026645 [4] |
| | 31. Seat Rail Pad- STDS2A1807 |

[1] Serial # 2A140801025... & higher

[2] Serial # 2A1510302680 & higher

[3] Serial # 2A1606300234 & higher

[4] Serial # 2A1606301542 & higher