


1025DS, 1025KS, 1025UK With Auxiliary Oxygen Port

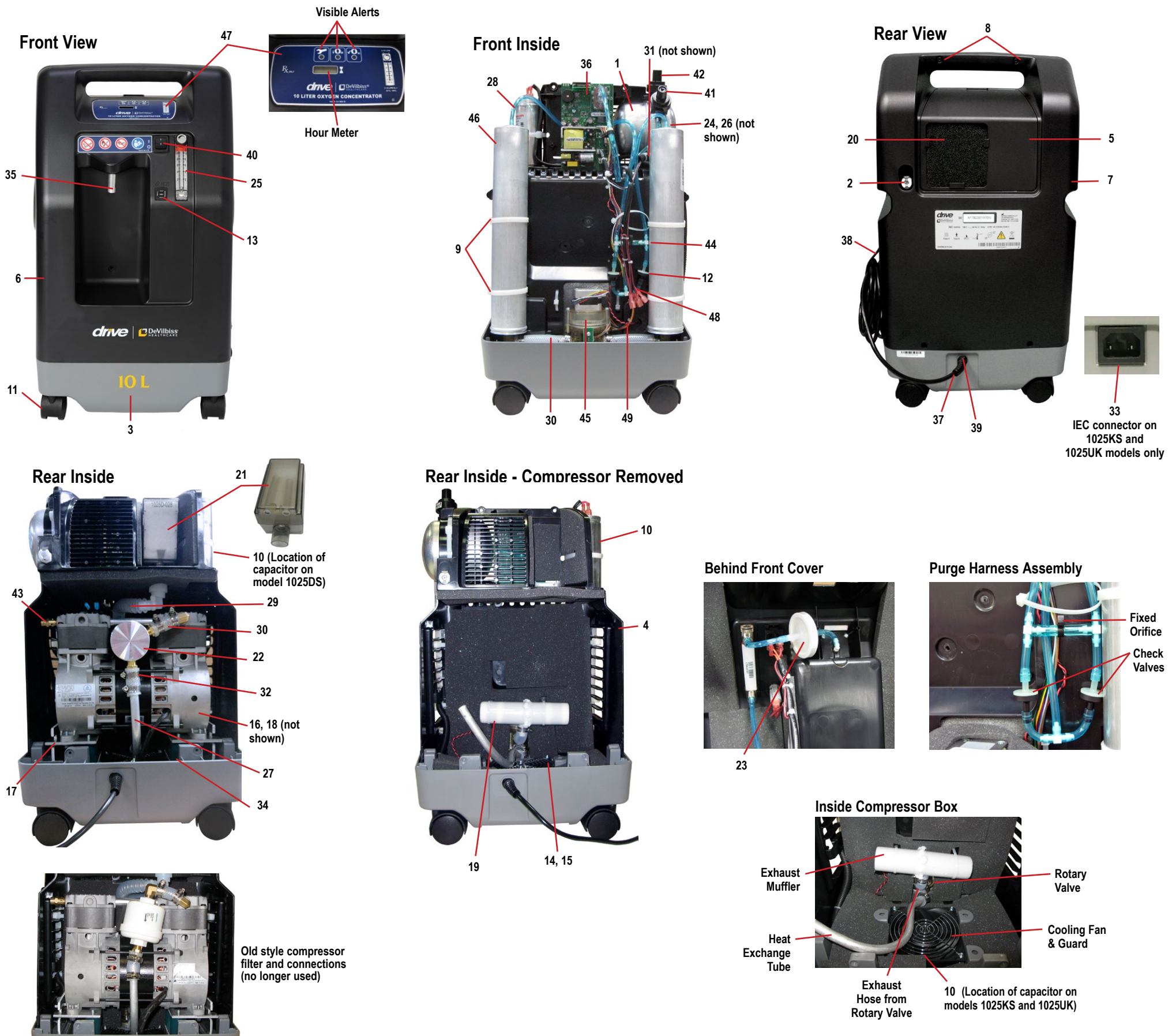

PART DESCRIPTION	1025DS	1025KS	1025UK
1 Accumulator Tank	1025D-610	1025D-610	1025D-610
2 Auxiliary Oxygen Port	525DD-635	525DD-635	525DD-635
Cabinet Parts:			
3 Base	1025D-603	1025K-603	1025K-603
4 Compressor Box w/ Foam	1025D-615	1025D-615	1025D-615
5 Filter Door	525DD-639	525DD-639	525DD-639
6 Front Cover	525DD-611	525DD-611	525DD-611
7 Rear Cover	525DD-612	525DD-612	525DD-612
8 Cabinet Screws	525DD-636	525DD-636	525DD-636
9 Cable Tie (Large)	505DZ-617	505DZ-617	505DZ-617
10 Capacitor (Motor Start/Run)	1025DS-616	1025KS-616	1025UK-616
11 Caster, Non-locking	501DZ-603	501DZ-603	501DZ-603
12 Bed Check Valve	PVO2D-607	PVO2D-607	PVO2D-607
13 Circuit Breaker	1025D-613	1025K-613	1025K-613
14 Cooling Fan	1025D-634	1025D-634	1025D-634
15 Cooling Fan Guard	N/A	N/A	N/A
16 Compressor	1025D-625	1025K-625	1025K-625
17 Compressor Mounting Plate Package	1025D-632	1025D-632	1025D-632
18 Compressor Rebuild Kit	1025D-643	1025K-643	1025K-643
19 Exhaust Muffler	1025D-705	1025D-705	1025D-705
Filters:			
20 Cabinet Air Filter	303DZ-605	303DZ-605	303DZ-605
21 Intake Bacteria Filter	1025D-605	1025D-605	1025D-605
22 Compressor Filter	1025D-682	1025D-682	1025D-682
23 Final Bacteria Filter	PV5LD-651	PV5LD-651	PV5LD-651
Fittings:			
24 Accumulator Tank "Tee" (not shown)	444-582	444-582	444-582
25 Flow Meter	1025D-607	1025D-607	1025D-607
26 Purge Manifold (not shown)	N/A	N/A	N/A
27 Heat Exchanger Tube	1025D-655	1025D-655	1025D-655

PART DESCRIPTION	1025DS	1025KS	1025UK
Hoses:			
28 1/8" ID (Blue) (4' Lg)	444-554	444-554	444-554
29 Compressor Intake Hose (Silicone)	1025D-657	1025D-657	1025D-657
30 1/2" ID Silicone Braided (2' LG)	505DZ-634	505DZ-634	505DZ-634
Hose Clamps:			
31 Plastic (1/4" ID Hose) (not shown)	N/A	N/A	N/A
32 Ladder Clamp (1/2" ID Hoses)	444-566	444-566	444-566
33 IEC Connector	N/A	N/A	N/A
34 Motor Mounts	1025D-609	1025D-609	1025D-609
35 Oxygen Outlet Port	525DD-606	525DD-606	525DD-606
36 PC Board	1025D-622	1025D-622	1025D-622
37 Power Cord w/ Strain Relief	PV5LD-618	VARIOUS	VARIOUS
38 Power Cord Strap	MC29D-657	MC29D-657	MC29D-657
39 Power Cord Strain Relief	505DZ-645	N/A	N/A
40 Power Switch	1025D-508	1025D-508	1025D-508
41 Pressure Regulator (8.5 PSI)	MC29D-612	MC29D-612	MC29D-612
42 Pressure Regulator (20 PSI)	1025D-612	1025D-612	1025D-612
43 Pressure Relief Valve	1025D-614	1025D-614	1025D-614
44 Purge Harness Assembly	1025D-617	1025D-617	1025D-617
45 Rotary Valve	515ADZ-702	515ADZ-702	515ADZ-702
46 Sieve Bed	1025D-619	1025D-619	1025D-619
47 Top Cover Label	1025D-631	1025D-631	1025D-631
Wire Harness:			
48 Main Harness	1025D-623	1025K-623	1025K-623
49 Rotary Valve Harness	525D-621	525D-621	525D-621