


SCOUT 3

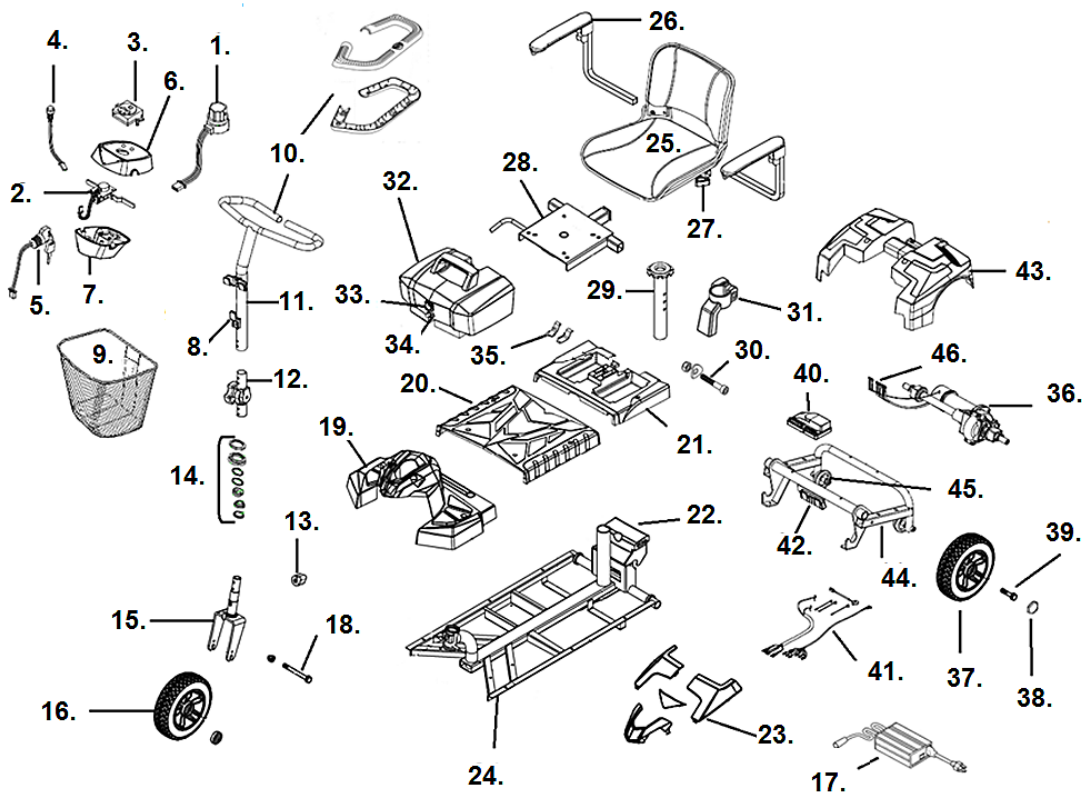

- | | |
|--|---|
| 1. Speed Dial- BC31001
Speed Dial Knob- LRM412103-11 | Right Armrest- BC35040R
Arm Pad- BC953014 |
| 2. Throttle Assembly Old- **
Throttle Assembly New- S12-069-00300
** Stock is gone use new style. | 27. Armrest Knob- S31052 |
| 3. Battery Gauge (4 LEDs)- SC31004
Battery Gauge (8 LEDs)- C11-076-00100 *
* After Serial # VP035868 | 28. Seat Plate- SC31023 |
| 4. Horn Button- SC31034 | 29. Seat Post- SC31018 *** |
| 5. Key Switch Assembly- BC31006
Key- BCKEY | 30. Seat Post Bolt- C07-071-00101 |
| 6. Upper Tiller Cover- SC31008 | 31. Battery Box Hold Down- SC31047 |
| 7. Lower Tiller Cover- SC31009 | 32. Battery Box & Batteries (12V 12ah)- SC31007
Battery Box Only (12V 12ah)- SC31006
Battery 12V 12ah (2 needed)- LRM402109
<u>EXT model</u> |
| 8. Basket Bracket- BC35051 | Battery Box & Batteries (12V 20ah)- SC31050
Battery Box Only (12V 20ah)- SC31051
Battery 12V 20ah (2 needed)- LRM412123 |
| 9. Basket- S35003-P | 33. Charger Socket & Harness- SC31031 |
| 10. Handgrip Foam- SC31002
Handgrip Plastic- SC31054 | 34. 20 Amp Circuit Breaker- BC20ACB |
| 11. Tiller- SC31001 | 35. Battery Contact- C12-060-00300 |
| 12. Tiller Swivel Assembly- SC31010 | 36. Motor/Brake/Transaxle Assembly- SC31053
Transaxle- BC350112
Motor- SC31036
Electro Magnetic Brake- SC31037 |
| 13. Tiller Lock- C08-069-01000 | 37. Rear Wheel Black- SC31003
Deluxe Rear Wheel- SC31003-D |
| 14. Tiller Bearing Set- BC31021 | 38. Hub Cap- SC31025 |
| 15. Fork- SC31038 | 39. Rear Axle Bolt Assembly- SC31049 |
| 16. Front Wheel Black- SC31043
Deluxe Front Wheel- SC31044-D | 40. Controller Curtis- SC31052
Controller Dynamic- C10-069-00100-3 *** |
| 17. Charger- LRM302111 | 41. Wire Harness Complete-
Curtis- SC31026
Dynamic- DST-018 *** |
| 18. Front Axle- SC31046 | 42. Contact Block Set w/Motor Harness- SC31033 |
| 19. Front Shroud- SC31039 | 43. Rear Shroud- SC31011 |
| 20. Foot Shroud- SC31024 | 44. Rear Frame- SC31012 |
| 21. Battery Box Shroud- SC31027 | 45. Anti-Tipper Wheel- S31026 |
| 22. Disconnect Handle Assembly- SC31019 | 46. Motor Wire Harness- SC31020 |
| 23. Red Decoration- SC31041
Deluxe Red Decoration- SC31041-D
Blue Decoration- SC31042
Deluxe Blue Decoration- SC31042-D | |
| 24. Front Frame- SC31040 | |
| 25. Black Seat- SC35045
Deluxe Two Tone Seat- SC31035
Deluxe Back Cover- SC31035-BC
Deluxe Seat Cover- SC31035-SC | |
| 26. Left Armrest- BC35040L | |

*** After Serial # VP142824

**** No longer available in chrome once stock is gone.