


VENTURA 4 & VENTURA 4 DLX (S/N # VP108386 & Higher)

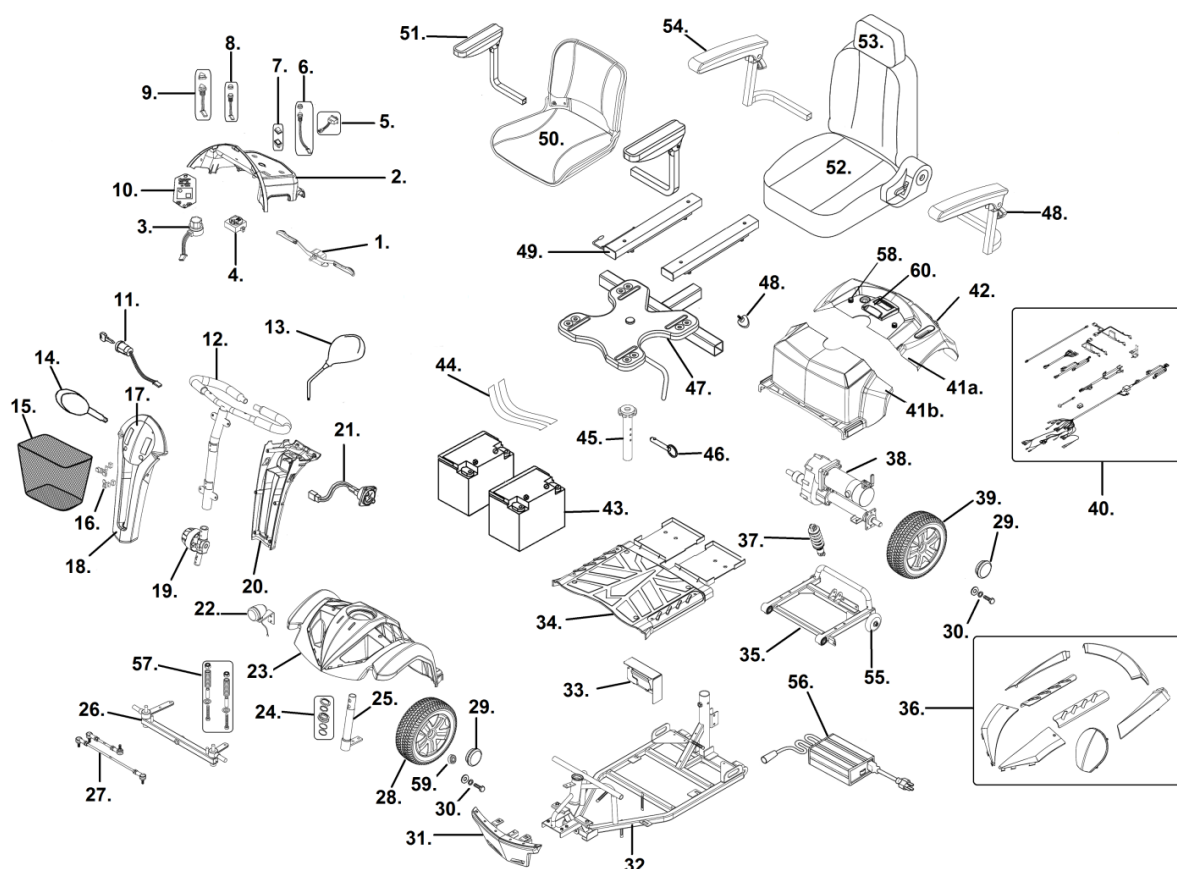

1. Forward-Reverse Assembly- VENT-01
Throttle Lever Only- VENT-87
2. Face Plate Assembly Folding Seat- VENT-04
Face Plate Assembly Capt. Seat- VENT-05
3. Speed Dial Assembly- S31001
Speed Dial Knob- LRM412103-11
4. Battery Gauge LED- C11-076-00100
Battery Gauge Analog- C11-033-00101
5. Turn Signal Switch Capt. Seat- VENT-72
6. Head Light Switch- S31122
Head light Switch Sub- GT8072W38
Headlight Switch Cover- GT8072W43
7. Fast/Slow Switch- GT8072W41
8. Horn Switch- S31012
Horn Switch Cover- GT8072W43
9. Flasher Switch- GT8072W32
10. Turn Signal Relay Capt. Seat- VENT-76
11. Key Switch- S41160
Key- S4KEY
12. Handlebar Assembly- VENT-03
13. Mirror Right- VENT-53
Mirror Left- VENT-54
14. Turn Signal Left- GT8072W35
Turn Signal Right- GT8072W36
Turn Signal Bulb- GT8072W53
15. Basket- S35003-PL
16. Basket Mounting Bracket- VENT-55
17. Front Decoration Red- VENT-61
Front Decoration Blue- VENT-62
18. Front Tiller Cover- VENT-06
19. Tiller Swivel Assembly- VENT-02
20. Rear Tiller Cover- VENT-07
21. Charger Socket- S31147
22. Head Light Assembly- VENT-44
23. Front Shroud- VENT-10
24. Stem Bearing Set- VENT-13
25. Steering Axle Assembly- VENT-27
26. Front Fork Assembly- VENT-23
27. Steering Rod Assembly- VENT-26
28. Front Wheel Assembly Solid- VENT-24
Front Wheel Assembly Pneumatic- VENT-56
Pneumatic Tire Only- VENT-59
Inner Tube Only- VENT-60
29. Hub Cap- VENT-70
30. Rear Axle Bolt- VENT-71
31. Front Bumper Assembly- VENT-14
32. Front Frame- VENT-47N
33. Controller- VENT-08
34. Middle Shroud- VENT-11N
35. Rear Frame- VENT-18N
36. Shroud Kit Red- S08-082-00118
Shroud Kit Blue- S08-082-00154
37. Shock Absorber- VENT-15N
38. Motor-Transaxle-Brake Assembly- VENT-19N
Transaxle- C09-082-00200
Motor- C09-082-00500
Motor Brush- SF5012
Electro Magnetic Brake- C09-067-00400
Free Wheel Lever & Hardware- VVENT-82
39. Rear Wheel Assembly Solid- VENT-28
Rear Wheel Assembly Pneumatic- VENT-57
Pneumatic Tire Only- VENT-59
Inner Tube Only- VENT-60
40. Wire Harness Fish-On Seat- S18-082-00100
Wire Harness Capt. Seat- S18-082-00200 [1]
(includes tiller and battery harness)
Battery Harness Only- S41158B
- 41a. Rear Shroud Folding Seat- VENT-12N
Rear Shroud Capt. Seat- VENT-69N
Rear Shroud Decoration Rd Lt- VENT-90N [2]
Rear Shroud Decoration Rd Rt- VENT-91N [2]
Rear Shroud Decoration BI Lt- VENT-92N [2]
Rear Shroud Decoration BI Rt- VENT-93N [2]
- 41b. Battery Shroud- C08-082-01100
Bat. Shroud Decoration Rd Lt- C08-082-01218 [2]
Bat. Shroud Decoration Rd Rt- C08-082-01318 [2]
Bat. Shroud Decoration BI Lt- C08-082-01254 [2]
Bat. Shroud Decoration BI Rt- C08-082-01354 [2]
42. Rear light Capt. Seat- VENT-66
Rear Left Turn light Capt. Seat- VENT-67
Rear Right Turn Light Capt. Seat- VENT-68
43. Battery 35 AH (2 needed)- LRA402208
44. Battery Strap- VENT-09
45. Seat Post- VENT-36 [3]
46. Seat Post Pin- S31152
47. Seat Plate Folding Seat- VENT-73
Seat Plate Capt. Seat- VENT-74
48. Arm Adjustment Knob- S31052
49. Seat Slides- VENT-65
50. Seat Assembly- VENT-41
51. Left Arm Assembly- VENT-29
Right Arm Assembly- VENT-30
Left Arm Pad- VENT-31
Right Arm Pad- VENT-32
52. Seat Assembly Capt. Seat 18"- VENT-42
Seat Assembly Capt. Seat 20"- VENT-43
53. Headrest Captain Seat 18"- SEC2010N
Headrest Captain Seat 20"- SF2010 [4]
54. Left Arm Assembly Capt. Seat- VENT-33
Right Arm Assembly Capt. Seat- VENT-34
Arm Pad Capt. Seat- VENT-35
55. Anti-Tip Wheels- VENT-16
56. Charger- LRA402210
57. Front Spring Set- VENT-77
58. Shroud Knob- VENT-86
59. Front Wheel Bearing- VENT-25
60. Frame Release Lever- C08-082-01400

[1] Serial # VP058688 and higher.

[2] When stock is gone these parts will only be available with part #36.

[3] No longer available in chrome, once chrome stock is gone.

[4] When stock is gone this part is no longer available.