**NITRO ELITE CF ROLLATOR
NITRO ELITE CF ROLLATOR
AMBULATEUR NITRO ELITE CF**

ITEM # RTL10266CF

driveTM

www.drivemedical.com

PARTS LIST

Item # RTL10266CF

WEIGHT CAPACITY: 300 LBS (136 KG)

- | | |
|-----------------------------------|------------------------|
| 1. Handgrip | 7. Seat |
| 2. Adjustable tubing handgrip | 8. Bag |
| 3. Rear reflector on handgrip | 9. Front wheel |
| 4. Brake lever | 10. Brake adjust screw |
| 5. Interlock button for handgrips | 11. Rear wheel |
| 6. Back Rest | 12. Cane Holder |

INTRODUCTION

The walking aid is suitable for a single user with limited mobility. The Nitro Elite is ideal for indoor and outdoor use.

NITRO INCLUDES:

1. Bag
2. Back Rest
3. Cane Holder
4. Operating Instructions


UNFOLDING THE NITRO

Position the Rollator on its wheels. Open the Rollator. Press the tubes attached to the seat downwards until they lock into position (figure 1).

REMOVING / ATTACHING THE CARRY BAG

To attach the bag, fold the Nitro slightly and affix the bag with the hook and loop fasteners to the tubes located on the front end of the seat frame (figure 2).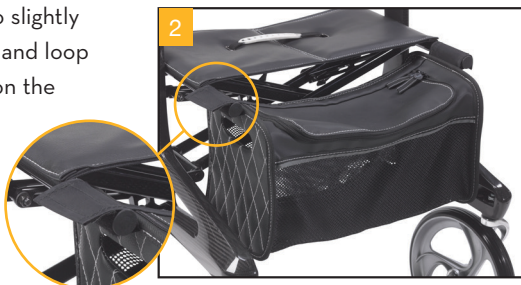

HEIGHT ADJUSTMENT

Press and hold down the white release buttons as shown in figure 3. Simultaneously move the handgrip up or down. When the desired height is reached, let go of the white release buttons and the handgrip will automatically lock into the next possible height position.

BRAKING / LOCKING

Pull both brake levers upwards to apply the brakes (see figure 4). Always apply brake pressure evenly.

To lock the brakes, ensure the Nitro is stationary, and press both brake levers down until they lock into position (see figure 7).

To release the brakes from the locked position, pull up on the brake levers.

Regularly test the brakes.

ADJUSTMENT OF BRAKES

To adjust the braking strength, adjust the brake by pressing the screw housing downwards (figure 6) and then turn left or right (figure 7).

SITTING

Position the Nitro on a flat surface. Lock the brakes (parking position). Sit straight up and do not lean backwards. (See figure 8).

DO NOT MOVE ROLLATOR WHILE SEATED.

FOLDING / TRANSPORT / STORAGE

To fold, pull up on the seat handle as shown in figure 9. The Nitro can be secured by pushing the two sides of the frame together until an audible click is heard signifying the lock is in place. Pull the handles apart to unlock.

SAFETY INSTRUCTIONS

- Use only as a walking aid!
- Use only on even and solid surfaces.
- Maximum user weight: 300 lbs.
- Regularly check the screws and fasteners on the Nitro and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 11 lbs.
- Do not expose the product to temperatures above 104°F for an extended period of time, as this can cause the Nitro parts to overheat and burn the user.
- Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the Rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.
- Replace tires when significantly worn or damaged.
- **DO NOT MOVE ROLLATOR WHILE SEATED.** This is a walking aid only and is not to be used as a transportation device.

CLEANING / MAINTENANCE

Clean regularly by hand with a damp cloth. Do not use aggressive detergents.

LIFETIME LIMITED WARRANTY

The frame of your Drive Medical product is warranted to be free of defects in material and workmanship for the lifetime of the original consumer purchaser. The brake handle and housing are covered for a period of 5 years of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

Warning: The items and packaging in this shipment contain chemicals known to the state of California to cause cancer, birth defects or other reproductive harm.

*This product meets or exceeds all federal safety and environmental regulations.

© 2016 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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© 2016 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.