


## 700-935

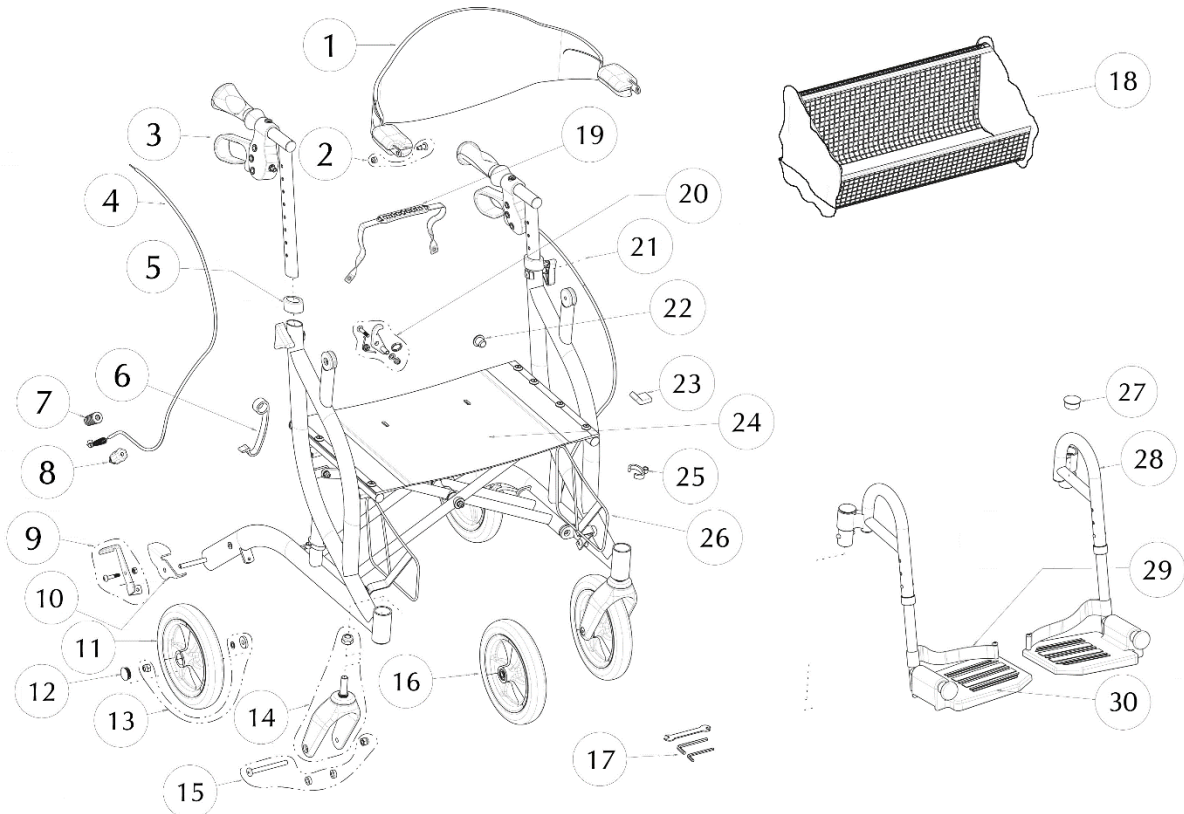

		REV1-0	R1-1	Qty Total	Qty Sold
1	Backrest with Armrests	B01-225	B01-225	1 EA	1 EA
2	Backrest Hardware	B01-861	B01-861	1 ST	1 ST
3	Handle Oval Bar & Brake Handle	B01-031BL	B01-031BL	1 PR	1 PR
4	Brake Cable & Sleeve Assy	B01-181	B01-181	1 PR	1 PR
5	Oval Anti-Rattle (2 each)	A01-211	A01-211	1 ST	1 ST
6	Fold Stopper Strap	A01-282	A01-282	1 EA	1 EA
7	Spring Sleeve, Black	A01-419	A01-419	2 EA	1 EA
8	Bottom Brake Adjustment Assy	B01-185	B01-185	2 ST	2 ST
9	Bottom Brake Assy, L&R	B01-159	C23-00027	1 ST	1 ST
10	Curb Climber right		C23-00117	1 EA	1 EA
11	Rear Wheel	B01-142	C23-00118	2 EA	1 EA
12	Hub Cap for Rear Wheel	A01-453	A01-453	2 EA	1 EA
13	Rear Wheel Hardware	B01-845	B01-845	2 ST	1 ST
14	Fork Assy.	B01-303	B01-303	2 EA	1 EA
15	Front Wheel Hardware	B01-832	B01-832	2 ST	1 ST
16	Front Wheel	B01-141	C23-00119	2 EA	1 EA
17	Tool Pack [1]	A01-712	A01-712	1 ST	1 ST
18	Soft Basket	A01-624	C23-00120	1 EA	1 EA
19	Seat Strap	A01-286	A01-286	1 EA	1 EA
20	Safety Lock	B01-881W	B01-881W	1 ST	1 ST
21	Triangular Knob (2 each)	A01-416	A01-416	1 ST	1 ST
22	Seat Caps (2 Front & 2 Rear)	B01-601	B01-601	1 ST	1 ST
23	Seat Support Brkt Cover (2 each)	A01-450	A01-450	2 ST	1 ST
24	Seat & Hardware	B01-282	B01-282	1 EA	1 EA
25	Oval Cable-Clip 2 each)	A01-244	A01-244	1 ST	1 ST
26	Mounting Brackets & Hardware	B01-873	B01-873	1 ST	1 ST
27	Front Cap Tubing, Black (2 each)	A01-455	A01-455	1 ST	1 ST
28	Footrest Assy L & R	B01-055	B01-055	1 PR	1 PR
29	Footrest Strap(2 each)	A01-291	A01-291	1 ST	1 ST
30	Footplate (pr)	B01-641	B01-641	1 PR	1 PR

[1] 4mm Allen Wrench, 5mm Allen Wrench & 8/10 mm wrench