


700-961, 700-959 Product Drawings

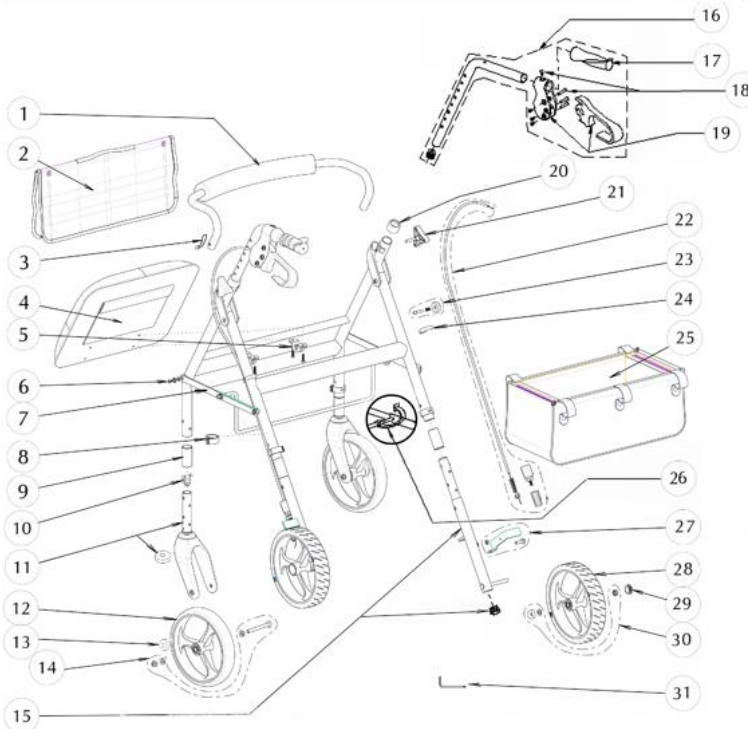

Revision	Rev 1.0	Rev 4.0	Rev 4.0	Rev 4.5	Rev 4.8	Rev 4.8	Rev 5.0	Rev 5.0	Rev 5.1	Qty	Sold	UOM
Date	8/1/2007	2/5/2008	11/26/2014	11/26/2014	5/7/2010	11/26/2014	1/26/2011	11/26/2014	6/22/2015	Per Unit	by	
1 Backrest Black	[2]	XXXX	B01-206BK	B01-206BK	XXXX	B01-206BK	XXXX	B01-206BK	XXXX	1	1	EA
Backrest Red	XXXX	[2]	XXXX	XXXX	[2]	XXXX	[2]	XXXX	[2]	1	1	EA
2 Saddle Bag	XXXX	A01-484	A01-484	A01-484	A01-484	A01-484	A01-484	A01-484	A01-484	1	1	EA
3 Push Button & Spring	B01-811	B01-811	B01-811	B01-811	B01-811	B01-811	B01-811	B01-811	B01-811	2	1	Set
4 Seat & Hardware	B01-270	B01-270	B01-270	B01-270	B01-270	B01-270	B01-270	B01-270	B01-270	1	1	EA
5 Seat Hardware	XXXX	B01-816	B01-816	B01-816	B01-816	B01-816	B01-816	B01-816	B01-816	1	1	Set
6 Folding Mechanism Hardware	XXXX	XXXX	[2]	[2]	B01-824	B01-824	B01-824	B01-824	B01-824	1	1	Set
7 Folding Mechanism	B01-075	B01-085	B01-085	B01-085	B01-085N	B01-085N	B01-085N	B01-085N	B01-085N	1	1	Set
8 E-Clip	A01-241	A01-241	A01-241	A01-241	A01-241	A01-241	A01-241	A01-241	A01-241	4	1	EA
9 Extension Tube	XXXX	A01-510 GR	A01-510GR	A01-510GR	A01-510GR	A01-510GR	A01-510GR	A01-510GR	A01-510GR	1	1	Set
10 Push Button	XXXX	XXXX	XXXX	A01-260	XXXX	XXXX	A01-260	A01-260	A01-260	4	1	PR
11 Fork	[2]	B01-124	B01-124	B01-124N	[2]	[2]	B01-124N	B01-124N	B01-124N	2	1	EA
Fork Stem Bearing	A01-301	A01-301	A01-301	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	4	1	EA
12 Caster	B01-133	B01-146	B01-146	B01-146	B01-146	B01-146	B01-146	B01-146	B01-146	2	1	EA
13 Caster Bearing	A01-302	A01-302	A01-302	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	8	1	EA
14 Caster Axle Assembly	B01-832	B01-832	B01-832	B01-832	B01-832	B01-832	B01-832	B01-832	B01-832	2	1	Set
15 Rear Leg	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	1	1	PR
End Plug	[2]	[2]	[2]	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	2	1	EA
16 Left Handlebar & Brake Handle Assy	B01-024L	B01-024L	B01-024L	B01-024L	B01-024L	B01-024L	B01-024L	B01-024L	B01-024L	1	1	EA
Right Handlebar & Brake Handle Assy	B01-024R	B01-024R	B01-024R	B01-024R	B01-024R	B01-024R	B01-024R	B01-024R	B01-024R	1	1	EA
17 Ortho-hand Grip Left & Right	[2]	[2]	[2]	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	1	1	PR
18 Hardware for Brake Handle	B01-851	B01-851	B01-851	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	2	1	Set
19 Brake Handle Assy	B01-156	B01-156	B01-156	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	2	1	Set
20 Anti-Rattle	[2]	[2]	[2]	[2]	[2]	[2]	[2]	[2]	[2]	2	1	EA
21 Knob	A01-415	A01-415	A01-415	A01-415	A01-415	A01-415	A01-415	A01-415	A01-415	2	1	Set
22 Brake Cable & Sleeve Assy	B01-157	B01-157	B01-157	B01-157	B01-157	B01-157	B01-157	B01-157	B01-181	2	1	Set
Brake Spring	[2]	[2]	[2]	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	2	1	EA
Spring Sleeve	A01-419	A01-419	A01-419	A01-419	A01-419	A01-419	A01-419	A01-419	XXXX	2	1	EA
23 Folding Mechanism Lock	XXXX	[2]	B01-090	B01-090	B01-091	B01-090	B01-091	B01-090	B01-090	1	1	EA
24 Cable Clip	XXXX	XXXX	XXXX	A01-243	A01-243	A01-243	A01-243	A01-243	A01-243	1	1	PR
25 Tote	A01-489	A01-483	A01-483	A01-483	A01-483	A01-483	A01-483	A01-483	A01-483	1	1	EA
26 Folding Latch	A01-480	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	1	1	EA
27 Wheel Brake Assy L & R	B01-159	B01-159	B01-159	B01-159	B01-159	B01-159	B01-159	B01-159	B01-159	1	1	Set
28 Rear Wheel	B01-136	B01-147	B01-147	B01-147	B01-147	B01-147	B01-147X	B01-147XN	B01-147XN	2	1	EA
29 Hub Cap	A01-453	A01-453	A01-453	A01-453	A01-453	A01-453	A01-453	A01-453	A01-453	2	1	EA
30 Rear Wheel Hardware	B01-845	B01-845	B01-845	B01-845	B01-845	B01-845	B01-845	B01-845	B01-845	2	1	Set
31 Tool Package	A01-701	A01-701	A01-701	[2]	[2]	[2]	[2]	[2]	[2]	1	1	PK

[1] When stock is gone the part is no longer available.
 [2] The part is no longer available.
 XXXX Not used on this diagram.


700-961, 700-959 Product Drawings

www.drivetech.info



700-959

www.drivetech.info

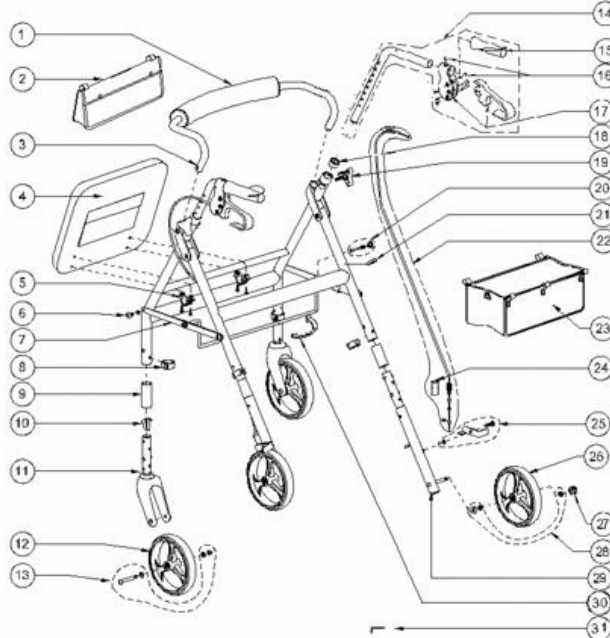

Revision	Rev 1.0	Rev 4.0	Rev 4.5	Rev 4.8	Rev 5.0	Rev 5.1	Qty	Sold	UOM
Date	8/1/2007	11/26/2014	11/26/2014	11/26/2014	11/26/2014	12/10/2014	Per Unit	by	
1 Curved Backrest, Black	[2]	[2]	[2]	[2]	[2]	[2]	1	1	EA
Curved Backrest, Blue	XXXX	XXXX	XXXX	XXXX	XXXX	XXXX	1	1	EA
2 Saddle Bag	XXXX	A01-484	A01-484	A01-484	A01-484	A01-484	1	1	EA
3 Push Button & Spring	B01-811	B01-811	B01-811	B01-811	B01-811	B01-811	2	1	Set
4 Seat with Hardware	B01-270	B01-270	B01-270	B01-270	B01-270	B01-270	1	1	EA
5 Seat Hardware	XXXX	B01-816	B01-816	B01-816	B01-816	B01-816	1	1	Set
6 Folding Mechanism Hardware	XXXX	[2]	[2]	B01-824	B01-824	B01-824	1	1	Set
7 Folding Mechanism	B01-075	B01-085	B01-085	B01-085N	B01-085N	B01-085N	1	1	Set
8 E-Clip	A01-241	A01-241	A01-241	A01-241	A01-241	A01-241	4	1	EA
9 Extension Tub (4 ea)	XXXX	A01-510PB	A01-510PB	A01-510PB	A01-510PB	A01-510PB	1	1	Set
10 Push Button (2 ea)	XXXX	XXXX	A01-260	XXXX	A01-260	A01-260	1	1	Set
11 Fork	[2]	B01-124	B01-124N	B01-124X[1]	B01-124N	B01-124N	2	1	EA
Fork Stem Bearing	A01-301	A01-301	XXXX	XXXX	XXXX	XXXX	4	1	EA
12 Caster	B01-133	B01-146	B01-146	B01-146	B01-146	B01-146	2	1	EA
Caster Bearing	A01-302	A01-302	XXXX	XXXX	XXXX	XXXX	8	1	EA
13 Caster Axle Assembly	B01-832	B01-832	B01-832	B01-832	B01-832	B01-832	2	1	Set
14 Left Handlebar & Brake Handle Assy	B01-024L	B01-024L	B01-024L	B01-024L	B01-024L	B01-024L	1	1	EA
Right Handlebar & Brake Handle Assy	B01-024R	B01-024R	B01-024R	B01-024R	B01-024R	B01-024R	1	1	EA
15 Ortho-hand Grip Left & Right	[2]	[2]	XXXX	XXXX	XXXX	XXXX	1	1	PR
16 Brake Handle Hardware	B01-851	B01-851	XXXX	XXXX	XXXX	XXXX	2	1	Set
17 Brake Handle Assy	B01-156	B01-156	XXXX	XXXX	XXXX	XXXX	2	1	Set
18 Anti-Rattle	[2]	[2]	[2]	[2]	[2]	[2]	2	1	EA
19 Knob	A01-415	A01-415	A01-415	A01-415	A01-415	A01-415	2	1	Set
20 Folding Mechanism Lock	XXXX	B01-090	B01-090	B01-090	B01-090	B01-090	1	1	Set
21 Cable Clip	XXXX	XXXX	A01-243	A01-243	A01-243	A01-243	1	1	PR
22 Brake Cable & Sleeve Assy	B01-157	B01-157	B01-157	B01-157	B01-157	B01-181	2	1	Set
23 Tote	A01-489	A01-483	A01-483	A01-483	A01-483	A01-483	1	1	EA
24 Brake Spring	[2]	[2]	XXXX	XXXX	XXXX	XXXX	2	1	EA
Spring Sleeve	A01-419	A01-419	A01-419	A01-419	A01-419	XXXX	2	1	EA
25 Wheel Brake Assy, Left & Right	B01-159	B01-159	B01-159	B01-159	B01-159	B01-159	1	1	PR
26 Rear Wheel	B01-136	B01-147	B01-147	B01-147	B01-147XN	B01-147XN	2	1	EA
27 Hub Cap	A01-453	A01-453	A01-453	A01-453	A01-453	A01-453	2	1	EA
28 Rear Wheel Hardware	B01-845	B01-845	B01-845	B01-845	B01-845	B01-845	2	1	Set
29 Rear Leg (pr)	XXXX	XXXX	XXXX	XXXX	XXXX	C23-00050	1	1	PR
29 End Plug	A01-421 [1]	A01-421 [1]	XXXX	XXXX	XXXX	XXXX	2	1	EA
30 Folding Lock Clip	A01-480	XXXX	XXXX	XXXX	XXXX	XXXX	1	1	EA
31 Tool Kit	A01-701	A01-701	[2]	[2]	[2]	[2]	1	1	PK

[1] When stock is gone the part is no longer available.

[2] The part is no longer available.

XXXX Not listed on this revision.