


10257

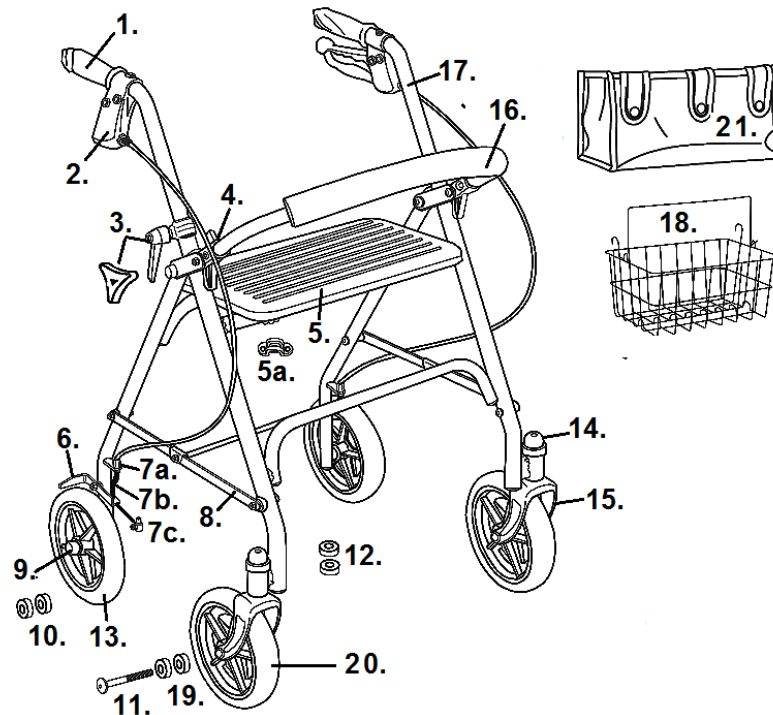

- 1. Hand Grip Left- 95032F260N15 [4]
Hand Grip Right- 95032F260N14 [4]
- 2. Hand Brake & Cable- 9502F1025701
- 3. Height Adj. Ratchet (pr)- 95032F260N10
Height Adj. Knob- 10208K [1]
- 4. Backrest Bracket (pr)- 9501S72603
- 5. Hard Seat & Brackets- [5]
Padded Seat & Brackets- 9502F1025712
- 5a. Hard Seat Brackets (pr)- [5]
Padded Seat Brackets (pr)- 9502F1025713
- 6. Wheel Lock Left- 9502F1025710
Wheel Lock Right- 9502F1025711
- 7a. Cable Adjuster- 9500001232
- 7b. Cable Spring- [5]
- 7c. Cable Fastener- 9500004101
- 8. Folding Hinge Assembly- 9502F1025707
- 9. Axle Cap- STDS001000
- 10. Rear Wheel Bearing- STDS1011
Rear Wheel Bearing (pr)- 1992M-19 [1]
- 11. Caster Axle- 9502F1025709
- 12. Fork Stem Bearing- STDS1060
- 13. Wheel Gray- 95032F260N09 [4]
- 14. Dust Cap (pr)- 95032F260N06
- 15. Fork Blue- 9502F1025708
Fork Red- 9502F1025708R
- 16. Backrest- 9502F1025705
- 17. Handlebar Only Left- 9502F1025715 [3] [4]
Handlebar Only Right- 9502F1025714 [3] [4]
Handlebar Assy. Left- 9503Z1025715 [1] [2]
Handlebar Assy. Right- 9503Z1025714 [1] [2]
Handlebar Assy. Left- 9503501025715 [7]
Handlebar Assy. Right- 9503501025714 [7]
- 18. Basket- 9502F1025706 [4]
- 19. Caster Bearing- STDS1011
Caster Bearing (pr)- 1992M-19 [1]
- 20. Caster Gray- 95032F260N09 [4]
Caster Black- [5]
Caster Black- 9503Z1025709 [1] [6]
- 21. Tote Bag- 9505W1026120
2023 and later- 9503Z1025716
- Not Shown Tray- [5]

[1] 3Z serial number.
 [2] Includes parts 1 and 2. Handle height is 35".
 [3] Handle height is 37".
 [4] When stock is gone the part is no longer available.
 [5] The part is no longer available.
 [6] Will fit on older versions.
 [7] 3Z serial number 350lbs model.