drive™

www.drivemedical.com

Rev.3.08.19.13

travel walker

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item # RTL10263KDR


EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

drive™

walker instructions

Easy “pull up” trigger release for opening and closing

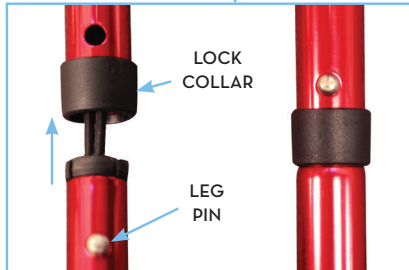IMPORTANT SAFETY NOTICE:

Your walker features flip up legs that can be attached to the frame of the walker for easy storage. Be sure that when the legs are turned down for use as a walker, the leg pin is inserted into the lock collar as shown above. Be sure to check the leg height adjustment hole position when setting up or folding for storage in the carry bag to be sure all hole adjustment are the same.

Rev.3.08.19.13

GLIDE SKIS

Be sure glide ski is centered on leg and facing forward with lip turned up at the front as shown.


custom walker adjustments

*With your arm extended down, handle should be at wrist height.
With your hand placed on the hand-grip, your elbow should be flexed 20° - 30°.*

CAUTION:

Be aware of the following:

- Electrical cords in way
- Liquids on floor
- Throw rugs
- Toys or debris
- Furniture
- Doors and door steps
- Gaps in floor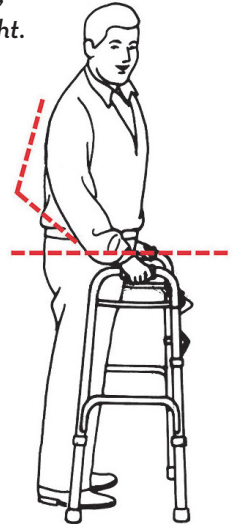


WARNING:

1. Use wheels on front legs only or on all four legs with appropriate rear wheel brakes.
2. Do not push one side of the walker ahead and then other side without lifting the walker first.
3. Use Drive accessories only. Use of other manufacturers' accessories can result in improper fit and risk fall and/or injury.
4. Do not use on stairs or escalators.

wheelchair to walker transfer


Making certain wheels on the wheelchair are locked lift your body by pushing down on the armrests and carefully transfer one hand at a time to the walker.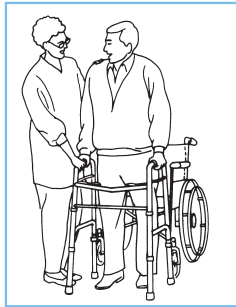

wheelchair to walker transfer

(with lower extremity injury to right leg or foot)

Making certain wheels on the wheelchair are locked, lift your body by pushing down on the armrests and carefully transferring first the left hand, then the right hand to the walker. (If injury to left leg or foot transfer the right hand first, followed by left.)

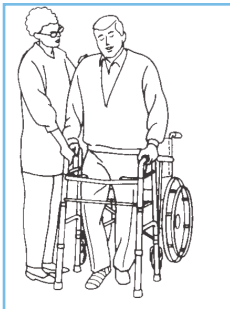three-point gait

(for walking with walker)

Step 1

Align the middle of your foot with the back legs of the walker.

Step 2

Slide the walker forward and place it comfortably in front of you.

Step 3

Carefully, walk forward one step at a time.

folding

1. Pull up on release lever.

2. Fold frame inside to close.

3. Repeat for opposite side.

4. To make the walker more compact for storage and transport, identify push pin above leg collar, depress pin and pull leg out of tube.

5. Fold legs up and insert into receptacle on frame. Repeat for remaining legs.

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lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2013 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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©2013 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2013 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.