
caution

Adult supervision is required at all times.
The straps and supports must be carefully
adjusted for the comfort and security of the child.
Child must be strapped in at all times.

assembly

Your Otter Bath Chair comes in one carton completely
assembled. This carton contains: Bath chair, head
cushion strap, positioning and leg straps.
If your shipment is incomplete, please call a Inspired by
Drive authorized dealer.

optional accessories

Tub Stand

Item # OT 8010, OT 8010XL

Shower Stand

Item # OT 8020 (NOT AVAILABLE FOR XL)

Soft Fabric Kit for OT 1000

Item # OT 1002, OT-1002-LL, OT-1002-BB

Soft Fabric Kit for OT 2000

Item # OT 2002, OT-2002-LL, OT-2002-BB

Soft Fabric Kit for OT 3000

Item # OT 3002, OT-3002-LL, OT-3002-BB

Soft Fabric Kit for OT 4000

Item # OT 4002, OT-4002-LL, OT-4002-BB

limited 1 year warranty

Your Inspired by Drive branded product is warranted
to be free of defects in materials and workmanship for
the lifetime of the product for the original consumer
purchaser.

This device was built to exacting standards and
carefully inspected prior to shipment. This Lifetime
Limited Warranty is an expression of our confidence
in the materials and workmanship of our products and
our assurance to the consumer of years of dependable
service.

This warranty does not cover device failure due to
owner misuse or negligence, or normal wear and
tear. The warranty does not extend to non-durable
components, such as rubber accessories, casters, and
grips, which are subject to normal wear and need
periodic replacement.

If you have a question about your Inspired by Drive
device or this warranty, please contact an authorized
provider.

⚠ WARNING: This product can expose you to chemicals,
including Styrene, which is known to the State of
California to cause cancer. For more information go to
www.P65Warnings.ca.gov.



2022 Medical Depot, Inc. All rights reserved.
Drive DeVilbiss Healthcare
99 Seaview Boulevard
Port Washington N.Y. 11050 USA
Made in China



otter bath chair

OT 1000

OT-1000SF-BB

OT-1000SF-LL

OT 2000

OT-2000SF-BB

OT-2000SF-LL

OT 3000

OT-3000SF-BB

OT-3000SF-LL



EU Authorized Representative
Y. Sung Handelsvertretung
Duesselthaler Str. 24
40211 Duesseldorf Germany

www.inspiredbydrive.com

maintenance and care

1. Inspect the chair periodically for cracks, breaks, loose or missing parts or malfunctions. Check the fabric for tears or worn areas. If a product needs service or replacement parts, contact an authorized Inspired by Drive dealer.
2. Wash chair and fabric with soap and water or 10% bleach. Do not use pine oil cleaners. Fabric and straps may be removed and machine-washed. Engage hook and loop before washing. Fabric cover may be washed on cool setting and drip-dried. Straps may be washed in warm water and dried on cool setting. Do not iron.

adjustments

See Figure 1

1. Head cushion strap height can be adjusted by opening strap in the back and positioning it at desired height. Securely engage hook and loop of strap once positioned. Slide head cushions (A) along the strap to adjust width.
2. Straps (D) can be adjusted as needed by inserting in any of the loops along the entire unit.
3. To adjust legs (E), pull crossbar (F) outward, rotate legs & push in to engage.
4. To adjust seat angle, pull on Uni-Bar under the seat (Fig 2) rotate seat to desired position and release Uni-Bar. To adjust back angle using Uni-Bar behind the Otter back (Fig 3).

caution

Adult supervision is required at all times. The straps and supports must be carefully adjusted for the comfort and security of the child. Child must be strapped in at all times.

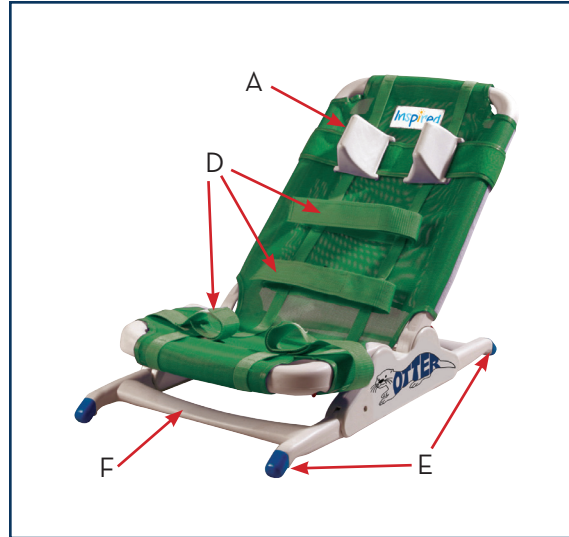


figure 1



figure 3



figure 2